


USER GUIDE

Access the key features from the Main Menu

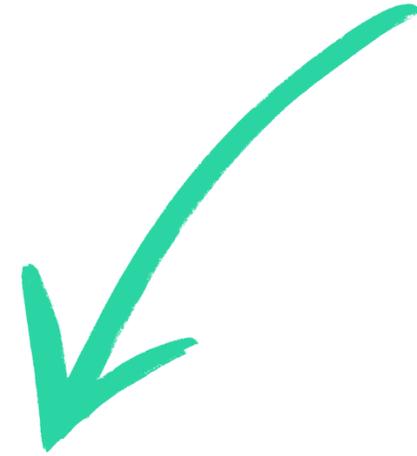

BELMONT (AJAX) LOG OUT

Main Menu Linda 



Sterilize 1
Add items to autoclave, start sterilization, certify instruments, record sterilization results.



Record BI Results 5
Record the results of Biological Indicator test.



Use Instruments
Record items to be used on a patient. Obtain control number here.



Check Status
Check the status of a particular item.

[GET REPORT](#)

**Start a new
sterilization cycle by
clicking on Sterilize
in the main menu.**



BELMONT (AJAX) LOG OUT

Main Menu Linda 



Sterilize 1
Add items to autoclave, start sterilization, certify instruments, record sterilization results.





Record BI Results 5
Record the results of Biological Indicator test.



Use Instruments
Record items to be used on a patient. Obtain control number here.



Check Status
Check the status of a particular item.

[GET REPORT](#)

← Main menu HOME LOG OUT


Sterilize Linda 

-  **MIDMARK M11 - 1** LOAD TIME : 44:52 
Contains C4 and Total 1 items
UNWRAPPED
-  **Tuttnauer EZ9Plus - 2** LOAD FINISHED 
Contains C4 and Total 1 items
WRAPPED INSTRUMENTS, POUCHES
-  **BRAVO 17V - 3** LOAD TIME : 42:37 
Contains total 1 items
BOWIE DICK
-  **MIDMARK M9 - 4** 
This autoclave is currently empty
POUCHES
-  **STATIM** 
NO-SERVICE



**Choose the autoclave
to run the
sterilization cycle.**



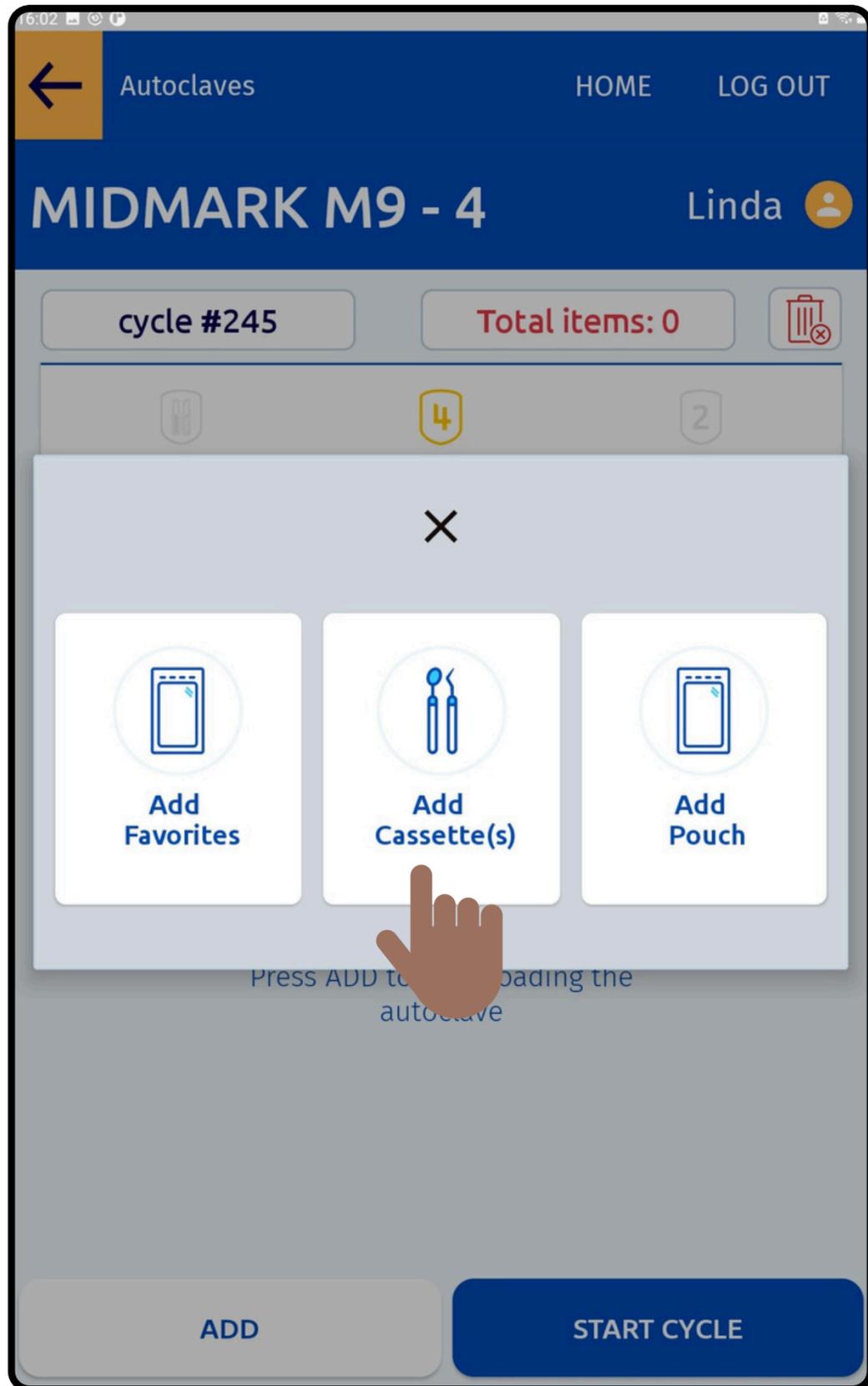

Add the Articles to the Autoclave based on the Category

Autoclaves HOME LOG OUT


Add Cassettes Linda

Load Type-4 Load BI Total items: 0

	Aeras 500 Elite,Lubricated	<input type="checkbox"/>
	Bridge Kit	<input type="checkbox"/>
	Hygiene Kit	<input type="checkbox"/>
	Implant	<input type="checkbox"/>
	Root Canal Kit	<input type="checkbox"/>
	Straumann Kit	<input type="checkbox"/>

Select the Item to Sterilize from the Customised List



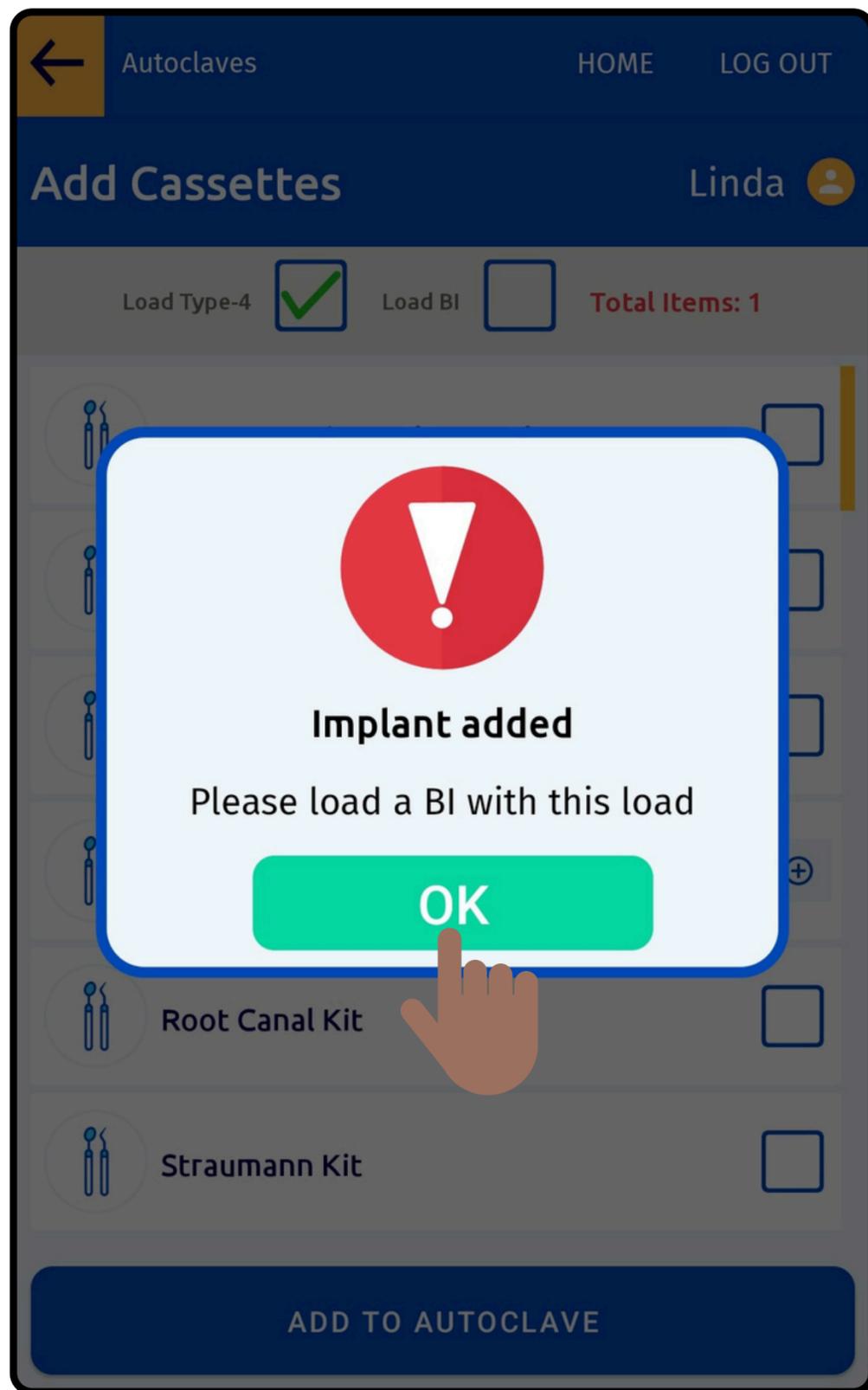

**Follow the Validations
provided**



← Autoclaves HOME LOG OUT

Add Cassettes Linda 


Load Type-4 Load BI Total Items: 1

	Aeras 500 Elite,Lubricate	<input type="checkbox"/>
	Bridge Kit	<input type="checkbox"/>
	Hygiene Kit	<input type="checkbox"/>
	Implant	<input type="text" value="1"/>
	Root Canal Kit	<input type="checkbox"/>
	Straumann Kit	<input type="checkbox"/>

ADD TO AUTOCLAVE


Add Biological Indicator to the Autoclave



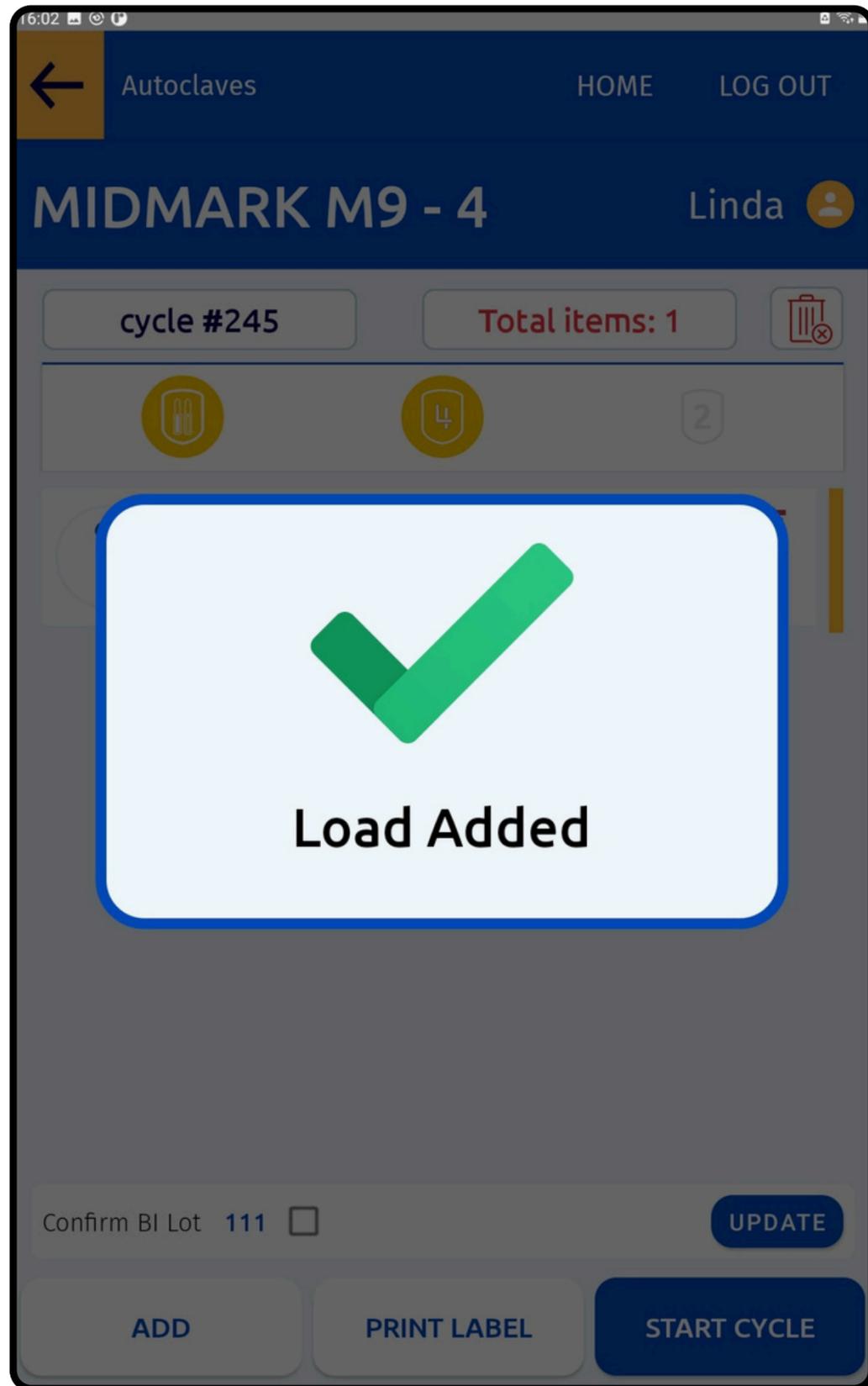



**Successfully Load the
Autoclave with the
selected items**



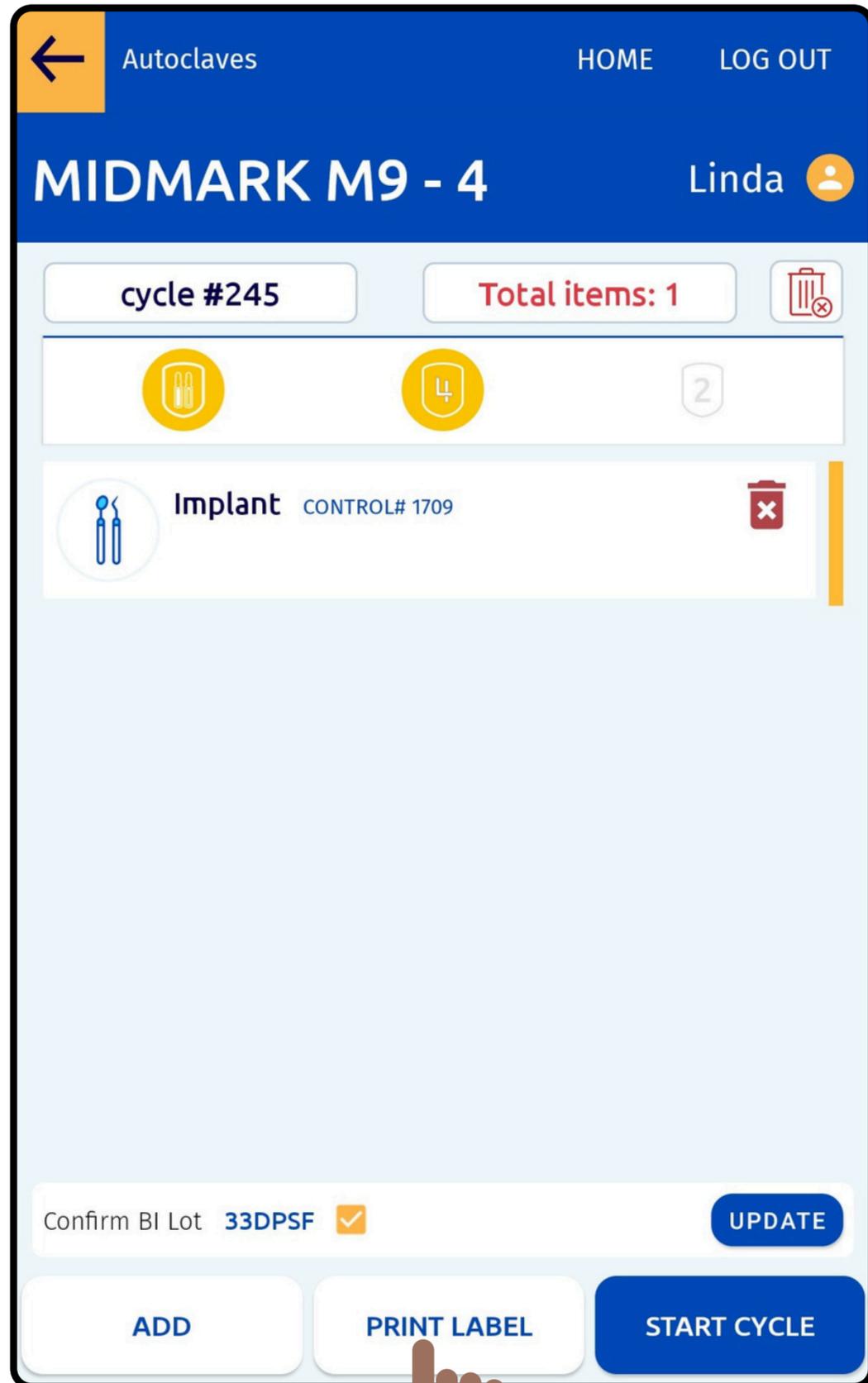


Click on Print Label to
print label(s)
for the Items added
to the Autoclave

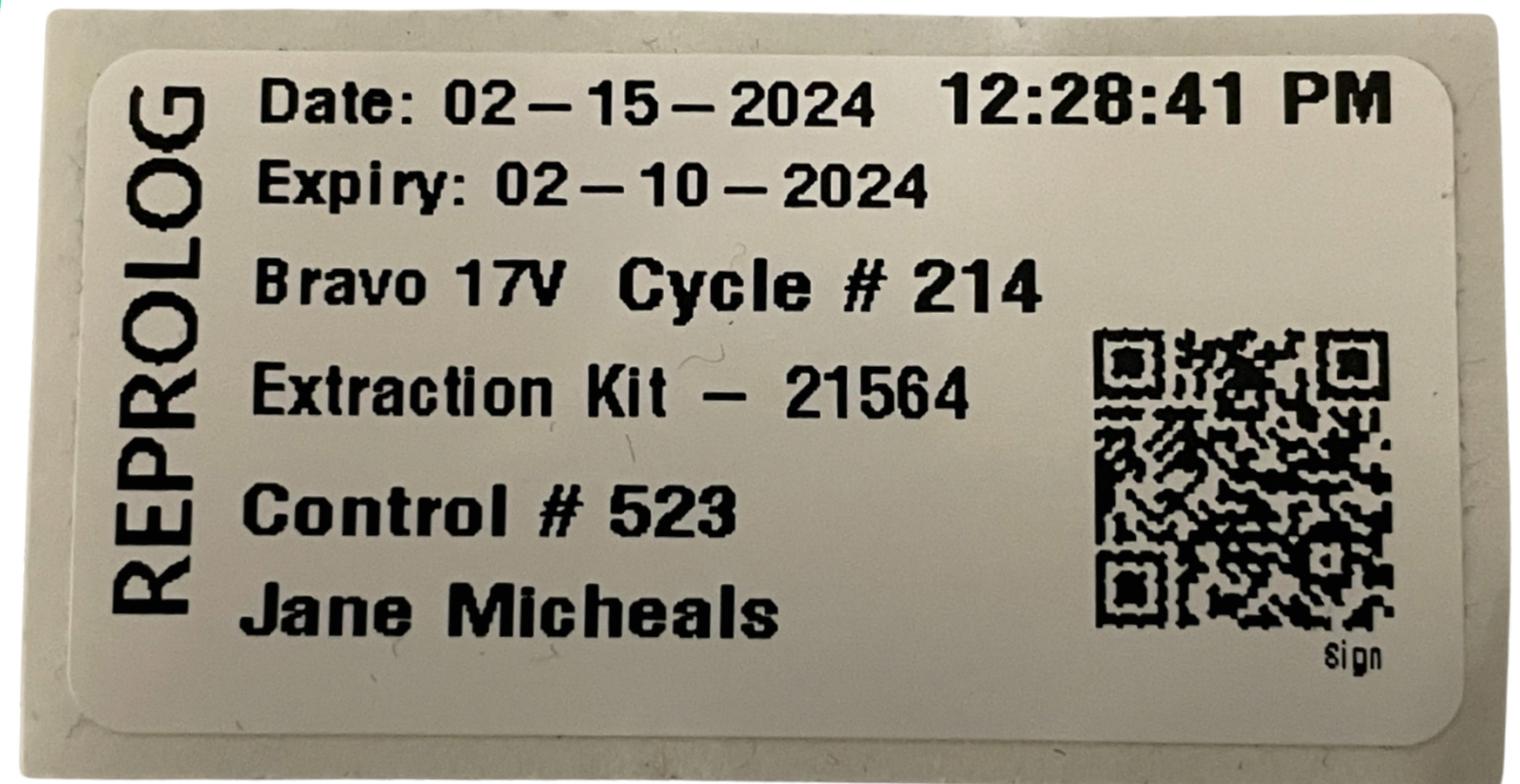

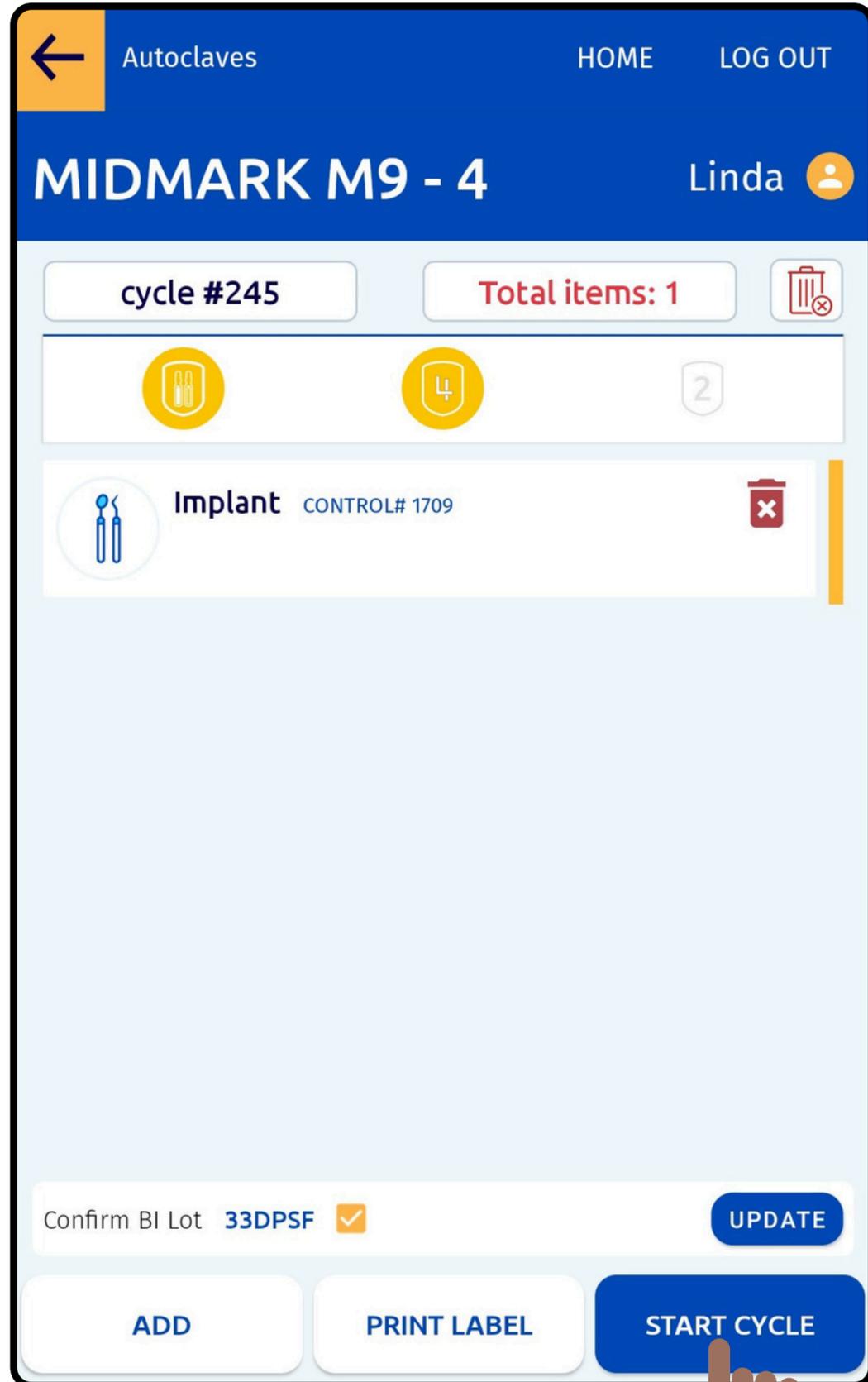

User can Remove or Add Items, Update BI Lot Number and Start a Cycle



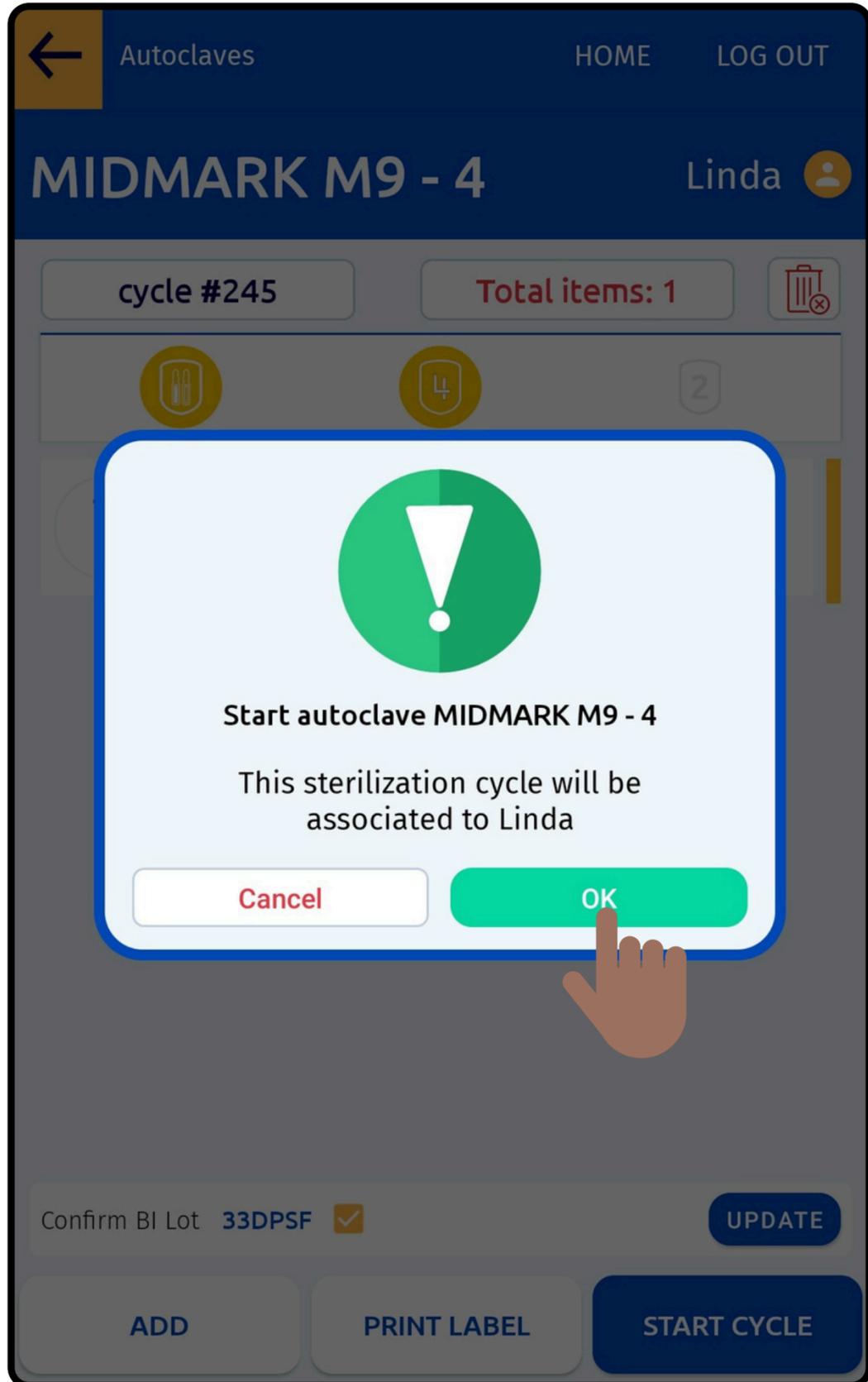

**Confirm the Cycle Details
for the Load to get started**




← Main menu HOME LOG OUT

Sterilize Linda 


-  **MIDMARK M11 - 1** LOAD TIME : 29:47 
Contains C4 and Total 1 items
UNWRAPPED 
-  **Tuttnauer EZ9Plus - 2** LOAD FINISHED 
Contains C4 and Total 1 items
WRAPPED INSTRUMENTS, POUCHES 
-  **BRAVO 17V - 3** LOAD TIME : 27:32 
Contains total 1 items
BOWIE DICK 
-  **MIDMARK M9 - 4** LOAD TIME : 44:46 
Contains C4, BI and total 1 items
HANDPIECES 
-  **STATIM 5000** 
NO-SERVICE 

View the Autoclave Load Details in Real-Time



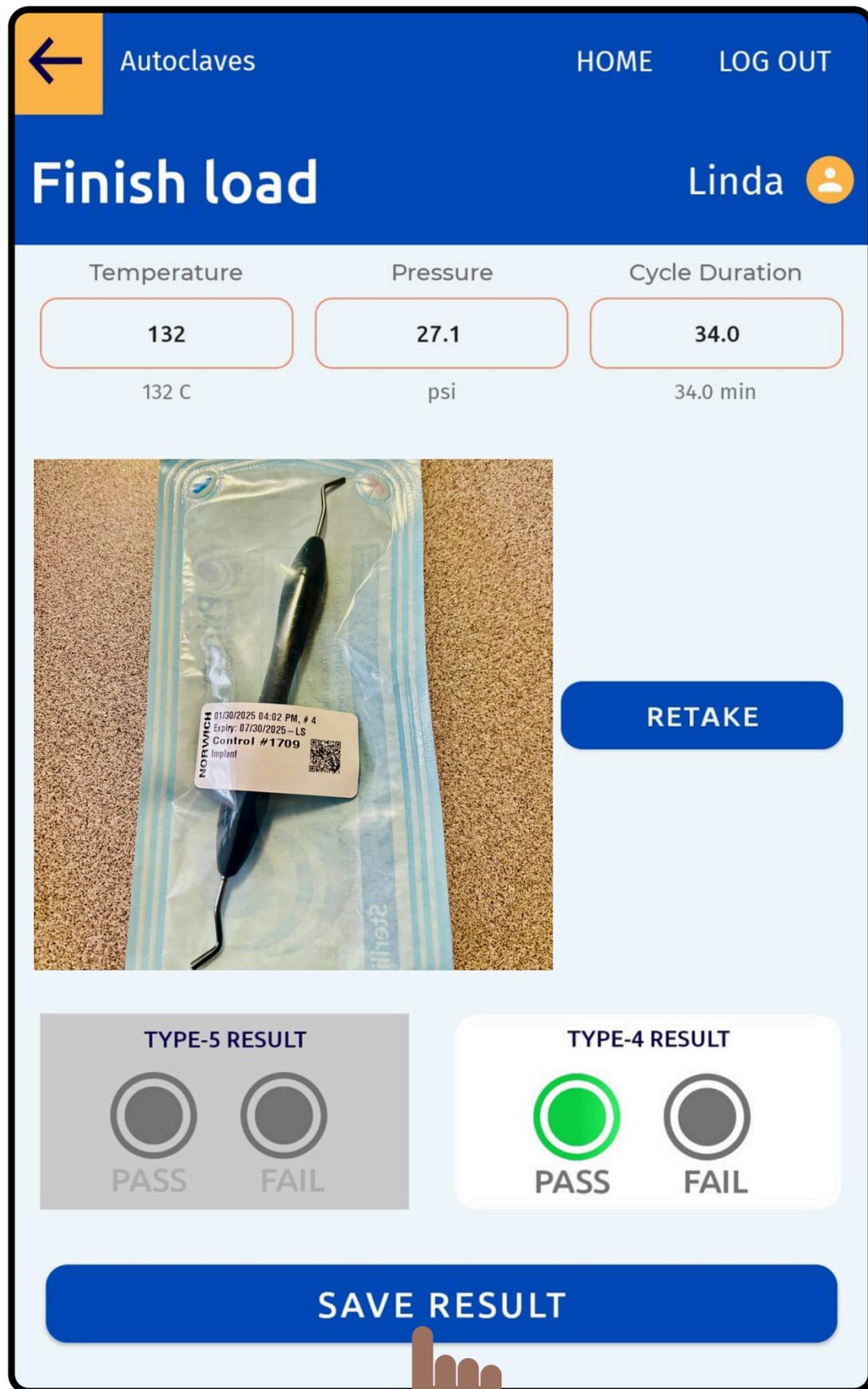

User can Add values for
Temperature,
Pressure,
and Cycle Duration
and
Capture the Image

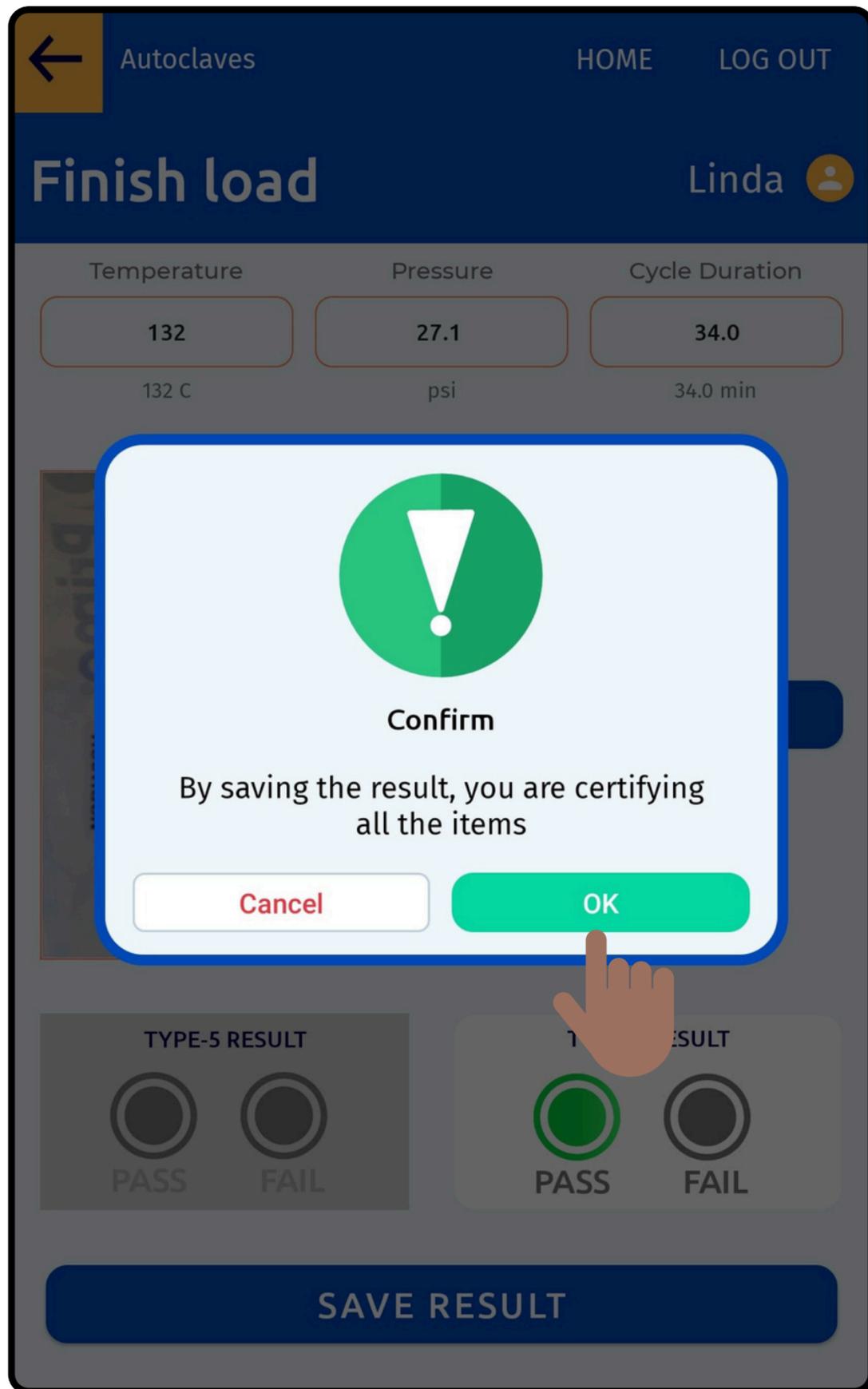

User can Finish the Load by adding the Result and Certifying the Items



Sterilize 1

Add items to autoclave, start sterilization, certify instruments, record sterilization results.



Record BI Results 5

Record the results of Biological Indicator test.



Use Instruments

Record items to be used on a patient. Obtain control number here.



Check Status

Check the status of a particular item.

GET REPORT

Select to Record Test Results of Biological Indicator




← Main Menu HOME LOG OUT

Select Autoclave Linda

STORE EXTERNAL SPORE TEST RESULTS

	STATIM 5000 Lot 111 cycle # 228 Unwrapped January-20-2025	C4 ●
	STATIM 5000 Lot 111 cycle # 230 Unwrapped January-20-2025	C4 ●
	Tuttnauer EZ9Plus - 2 Lot 111 cycle # 221 Wrapped Instruments, Pouches January-20-2025	C4 ●
	MIDMARK M9 - 4 Lot 111 cycle # 244 Pouches January-20-2025	C4 ●
	MIDMARK M11 - 1 Lot 111 cycle # 200 Packs January-20-2025	C4 ●

Click on the Autoclave name to select and Record BI Results

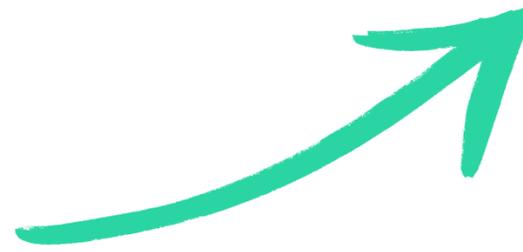

← Main menu HOME LOG OUT

BI Result Linda

AUTOCLAVE CYCLE # BI LOT # DATE
STATIM 5000 228 111 January-20-2025



RETAKE

CONTROL VIAL

GROWTH NO GROWTH

TEST VIAL

GROWTH NO GROWTH

ADDITIONAL COMMENTS


Add Notes

SAVE BI RESULT

Save the BI Result by capturing an image and recording the growth of Control and Test vials



Click on Use Instruments to Record and Return Items

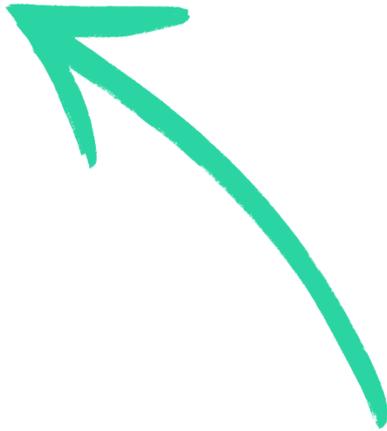

BELMONT (AJAX) LOG OUT

Main Menu Linda 



Sterilize 2 2

Add items to autoclave, start sterilization, certify instruments, record sterilization results.



Record BI Results

Record the results of Biological Indicator test.



Use Instruments

Record items to be used on a patient. Obtain control number here.






Check Status


Check the status of a particular item.

[GET REPORT](#)

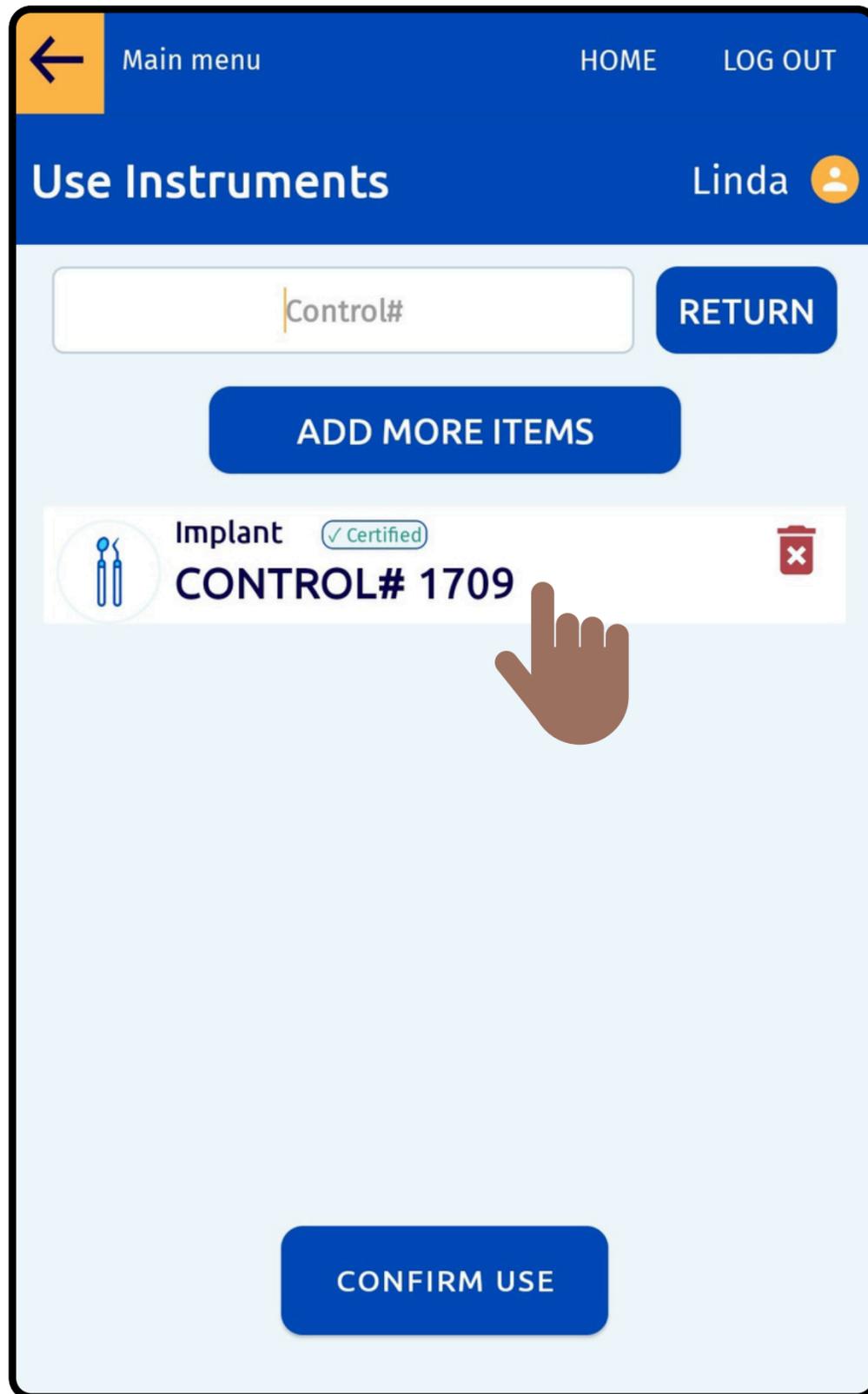

**Enter the Control Number
to add Items
for Use or Return**



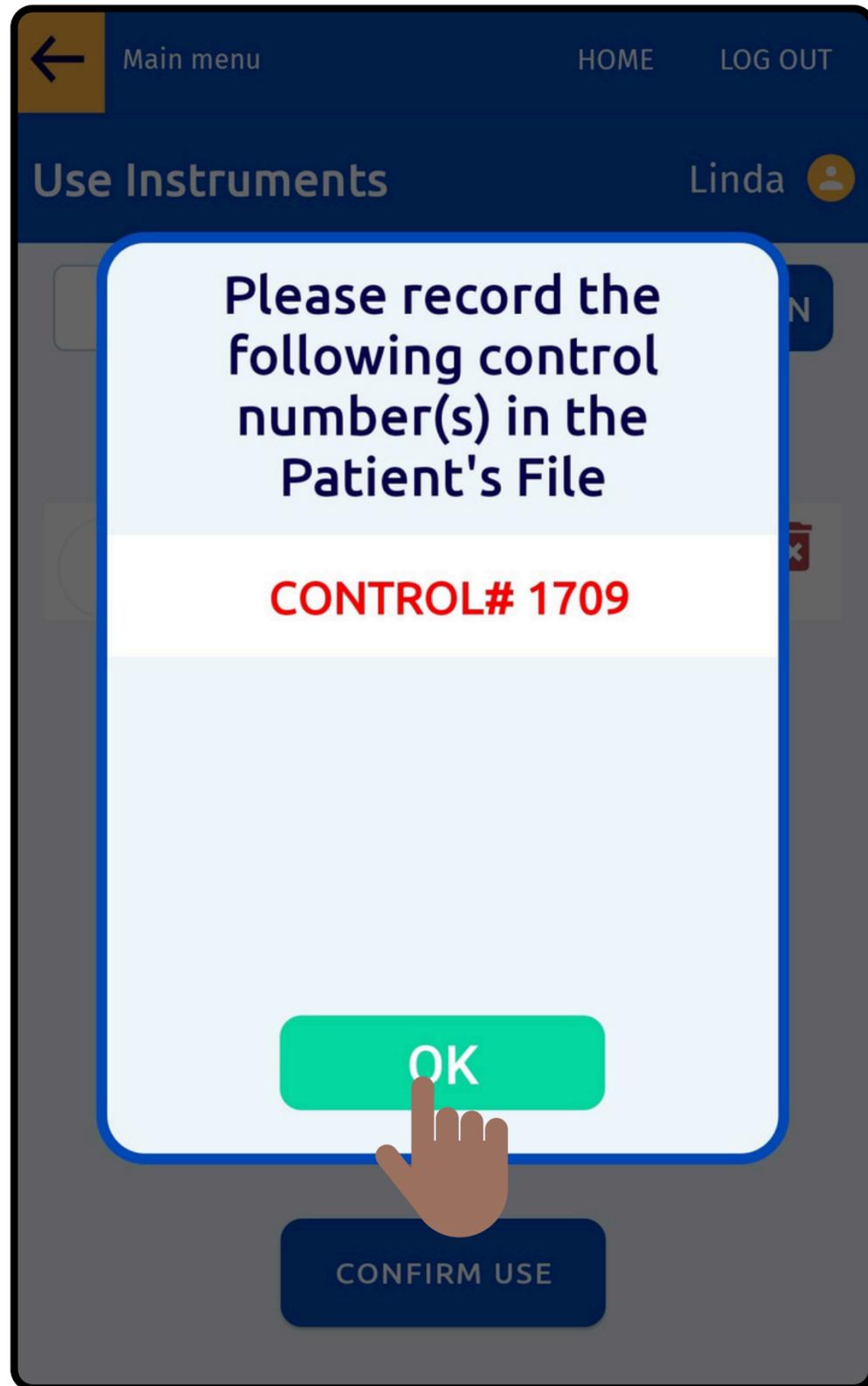

**Click on OK to
Confirm and Record
the Control Numbers
into the Patient File**





Sterilize 2 2

Add items to autoclave, start sterilization, certify instruments, record sterilization results.



Record BI Results

Record the results of Biological Indicator test.



Use Instruments

Record items to be used on a patient. Obtain control number here.



Check Status


Check the status of a particular item.




GET REPORT

Select Check Status to view the status of Items




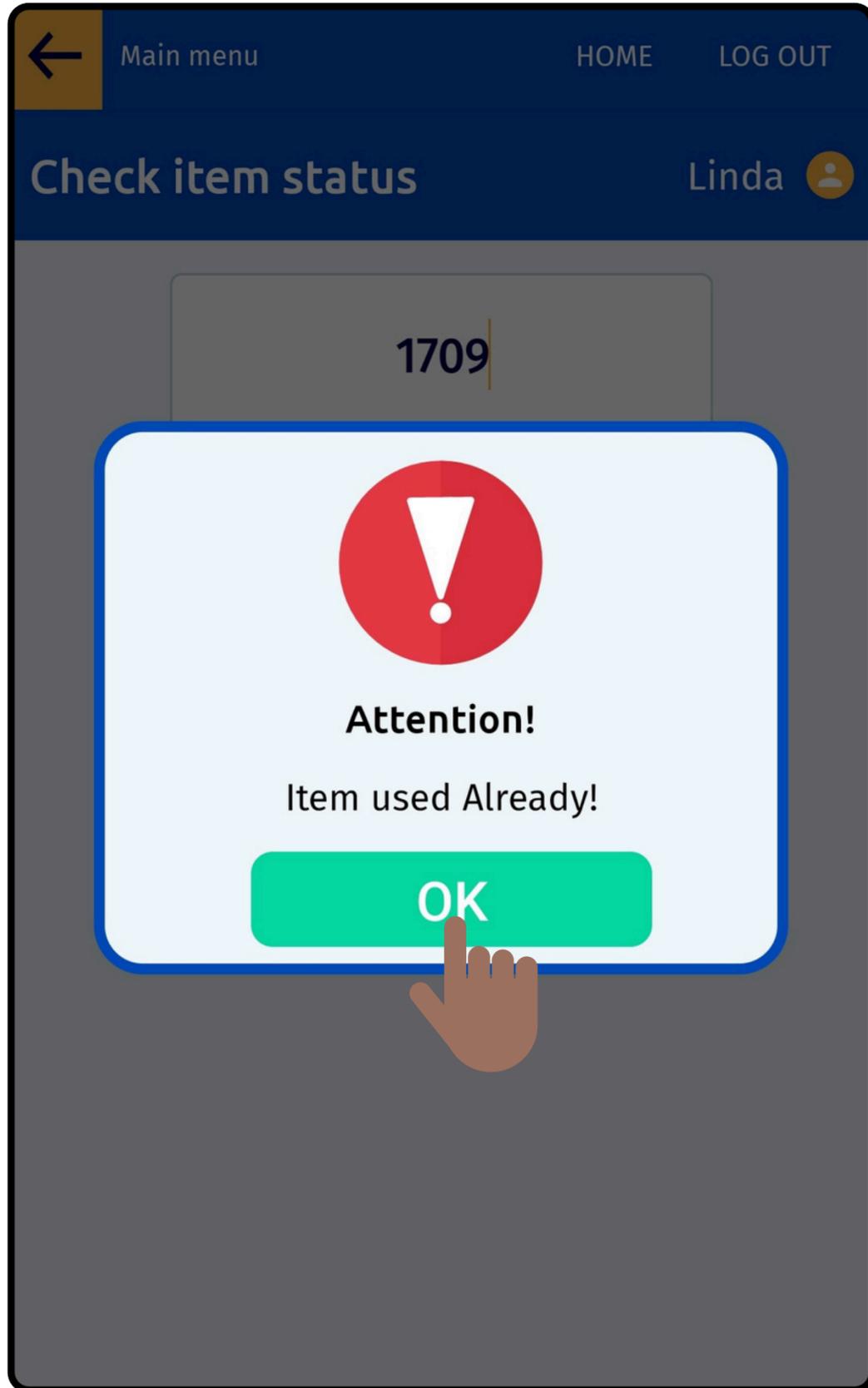

**Enter the Control Number
to view the Current Status
of that Item**

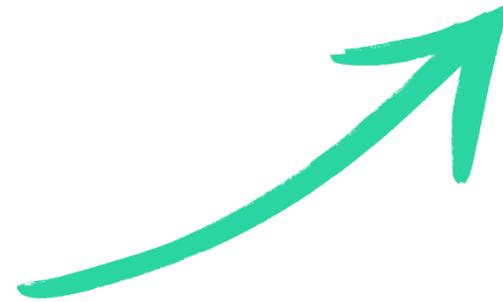

Select the Get Report button to View and Email the Details of an Item

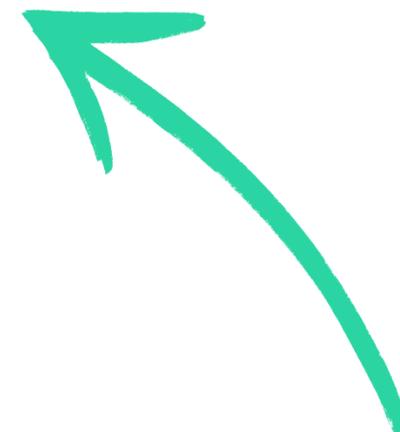

BELMONT (AJAX) LOG OUT

Main Menu Linda

Sterilize 2 2

Add items to autoclave, start sterilization, certify instruments, record sterilization results.

Record BI Results

Record the results of Biological Indicator test.

Use Instruments

Record items to be used on a patient. Obtain control number here.

Check Status

Check the status of a particular item.

GET REPORT

Enter the Control Number and Click Ok



BELMONT (AJAX) LOG OUT

Main Menu Linda 

Please enter control

CANCEL **OK**

Use Instruments
Record items to be used on a patient. Obtain control number here.

Check Status
Check the status of a particular item.

GET REPORT

CONTROL# 1709

Article Name Implant

Loaded By Linda Smith

Sterilized On 30-January-2025

Autoclave Name MIDMARK M9 - 4

Started By Linda Smith

cycle # 245

Used By Linda Smith

Used On 2025-01-30

GO BACK



EMAIL REPORT



The user can View the recorded details and Email the report as needed



SCAN ME

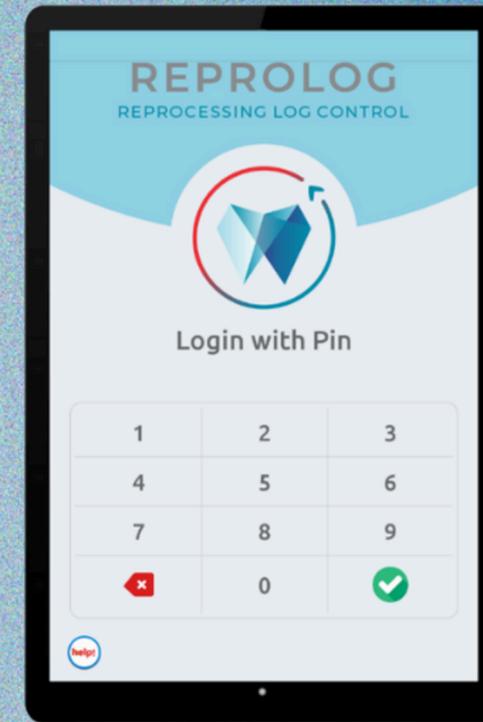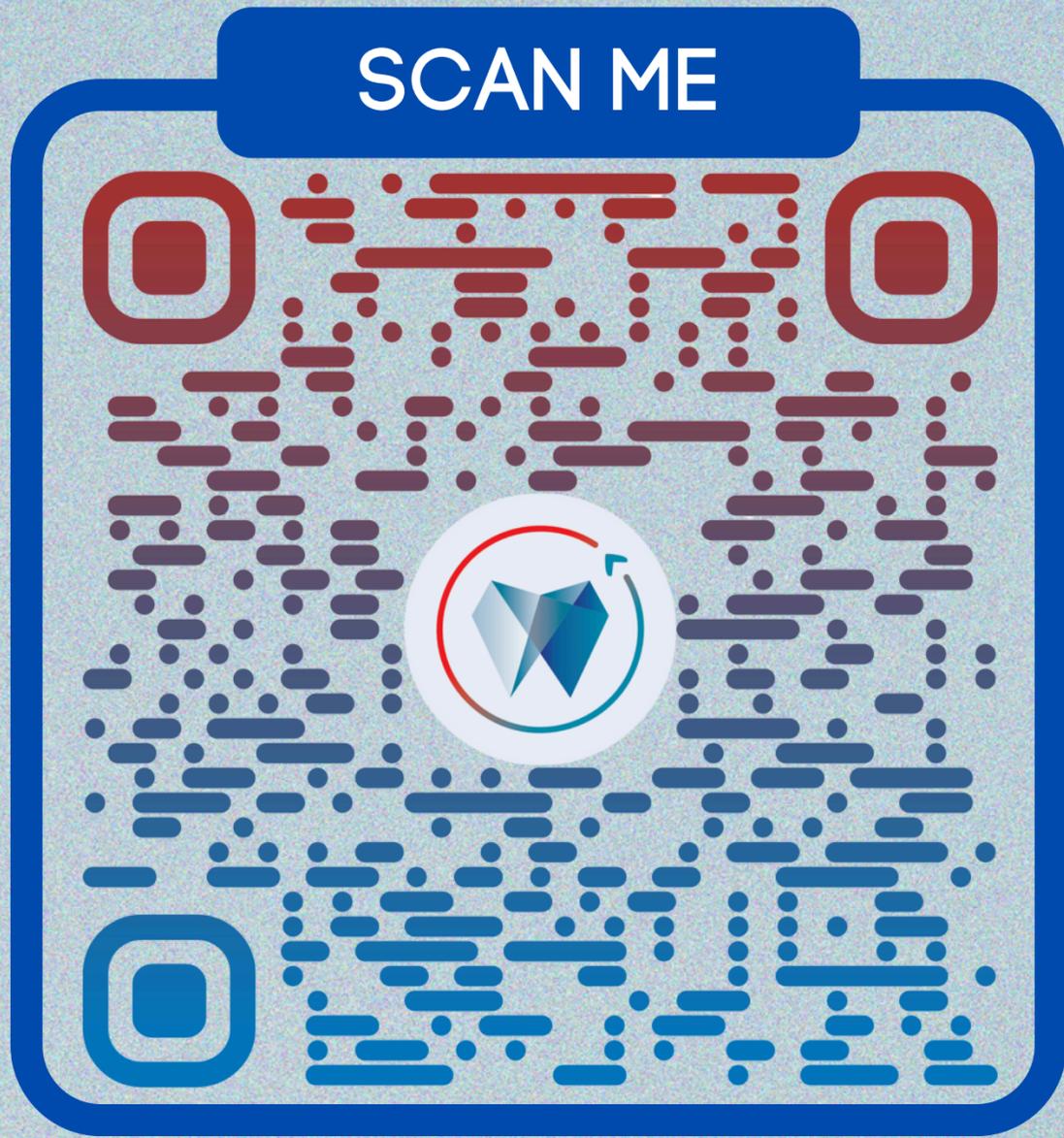

PHONE

+1.416-423-2900



EMAIL

info@reprolog.net

