



# USER GUIDE




# Access the key features from the Main Menu

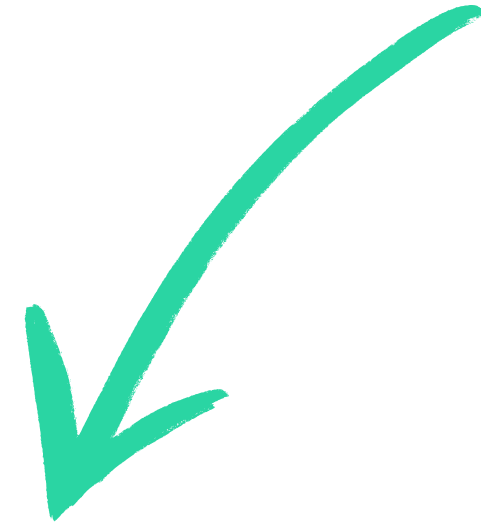


BELMONT ( AJAX ) LOG OUT


**Main Menu** Linda 




**Sterilize 1**  
Add items to autoclave, start sterilization, certify instruments, record sterilization results.



**Record BI Results 5**  
Record the results of Biological Indicator test.



**Use Instruments**  
Record items to be used on a patient. Obtain control number here.




**Check Status**  
Check the status of a particular item.


[GET REPORT](#)




**Start a new  
sterilization cycle by  
clicking on Sterilize  
in the main menu.**

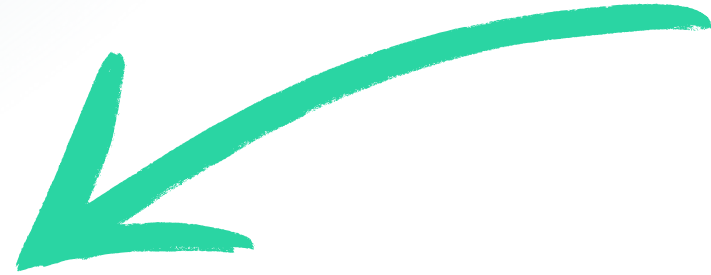



BELMONT ( AJAX ) LOG OUT

**Main Menu** Linda 




**Sterilize 1**  
Add items to autoclave, start sterilization, certify instruments, record sterilization results.






**Record BI Results 5**  
Record the results of Biological Indicator test.



**Use Instruments**  
Record items to be used on a patient. Obtain control number here.














**Check Status**  
Check the status of a particular item.


[GET REPORT](#)




← Main menu HOME LOG OUT

**Sterilize** Linda 


-  **MIDMARK M11 - 1** LOAD TIME : 44:52   
Contains C4 and Total 1 items  
UNWRAPPED
-  **Tuttnauer EZ9Plus - 2** LOAD FINISHED   
Contains C4 and Total 1 items  
WRAPPED INSTRUMENTS, POUCHES
-  **BRAVO 17V - 3** LOAD TIME : 42:37   
Contains total 1 items  
BOWIE DICK
-  **MIDMARK M9 - 4**   
This autoclave is currently empty  
POUCHES
-  **STATIM**   
NO-SERVICE

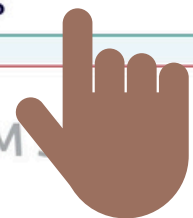


**Choose the autoclave  
to run the  
sterilization cycle.**

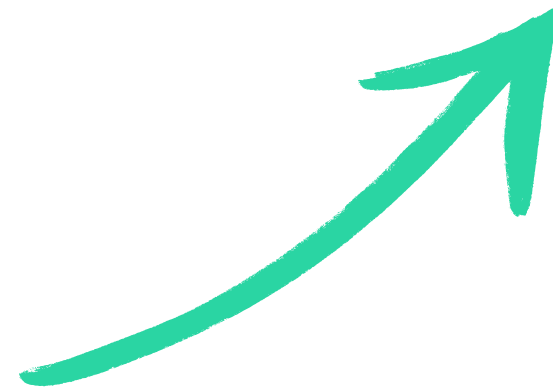



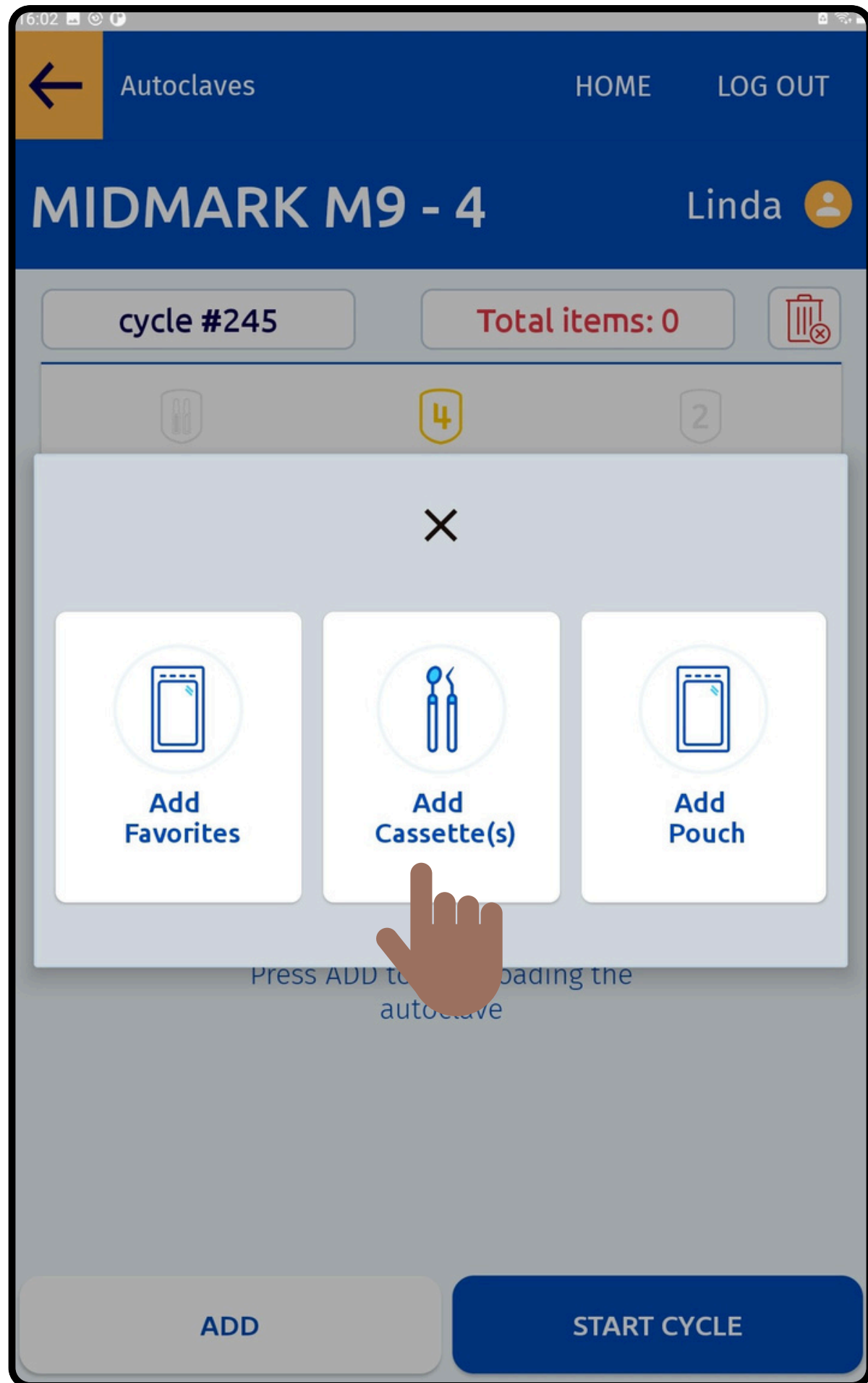

**Add the Articles to the Autoclave based on the Category**

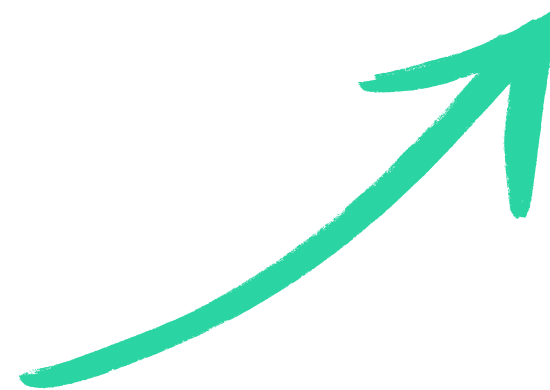










Autoclaves HOME LOG OUT

Add Cassettes Linda


Load Type-4  Load BI  Total items: 0

	Aeras 500 Elite,Lubricated	<input type="checkbox"/>
	Bridge Kit	<input type="checkbox"/>
	Hygiene Kit	<input type="checkbox"/>
	Implant	<input type="checkbox"/>
	Root Canal Kit	<input type="checkbox"/>
	Straumann Kit	<input type="checkbox"/>


Select the Item to Sterilize from the Customised List

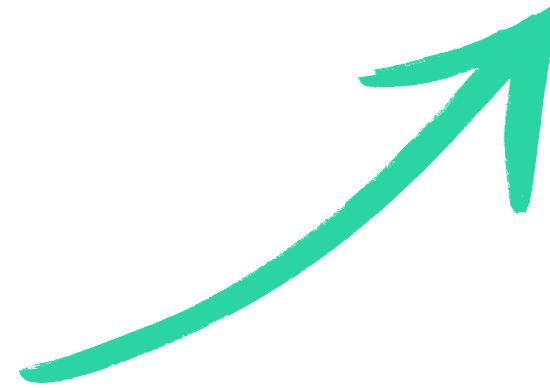



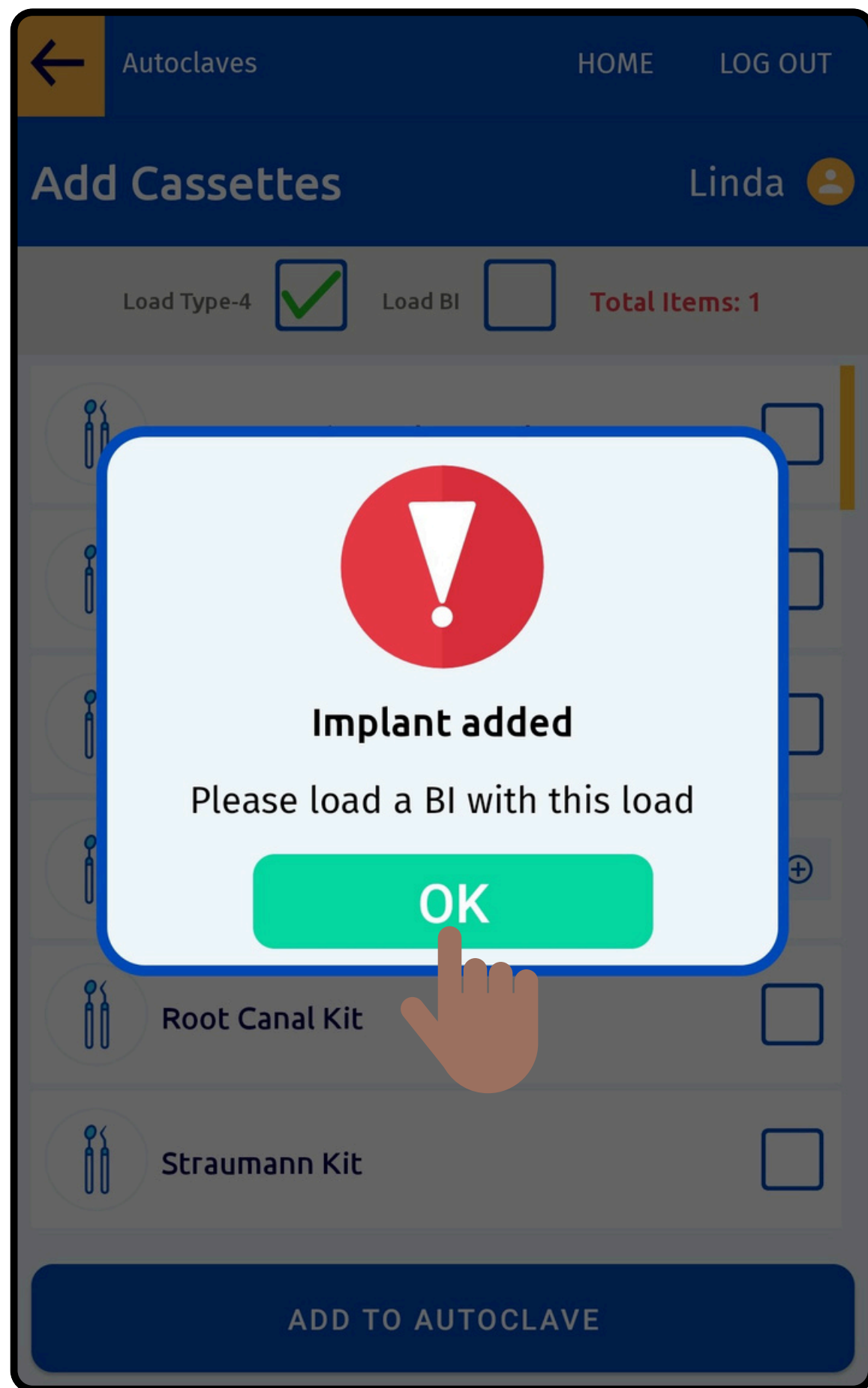


**Follow the Validations  
provided**

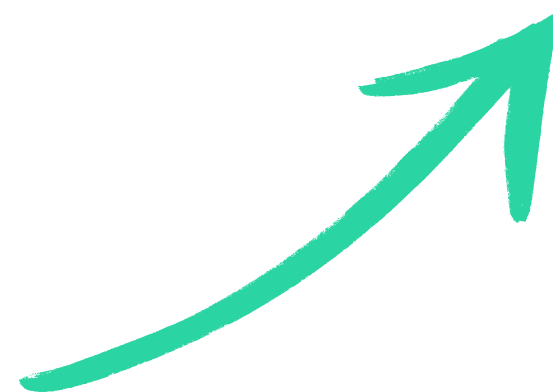









← Autoclaves HOME LOG OUT


Add Cassettes Linda 

Load Type-4  Load BI  Total Items: 1


	Aeras 500 Elite,Lubricate	<input type="checkbox"/>
	Bridge Kit	<input type="checkbox"/>
	Hygiene Kit	<input type="checkbox"/>
	Implant	<input type="checkbox"/> <input type="button" value="−"/> 1 <input type="button" value="+"/>
	Root Canal Kit	<input type="checkbox"/>
	Straumann Kit	<input type="checkbox"/>

ADD TO AUTOCLAVE


# Add Biological Indicator to the Autoclave

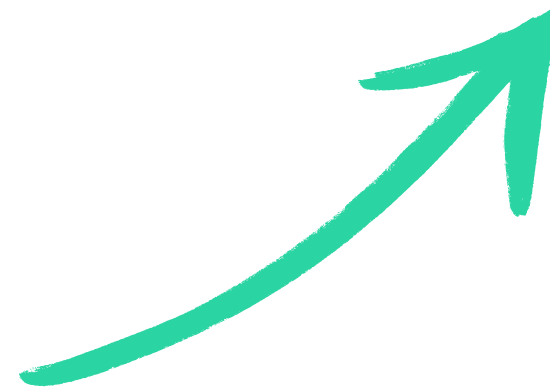




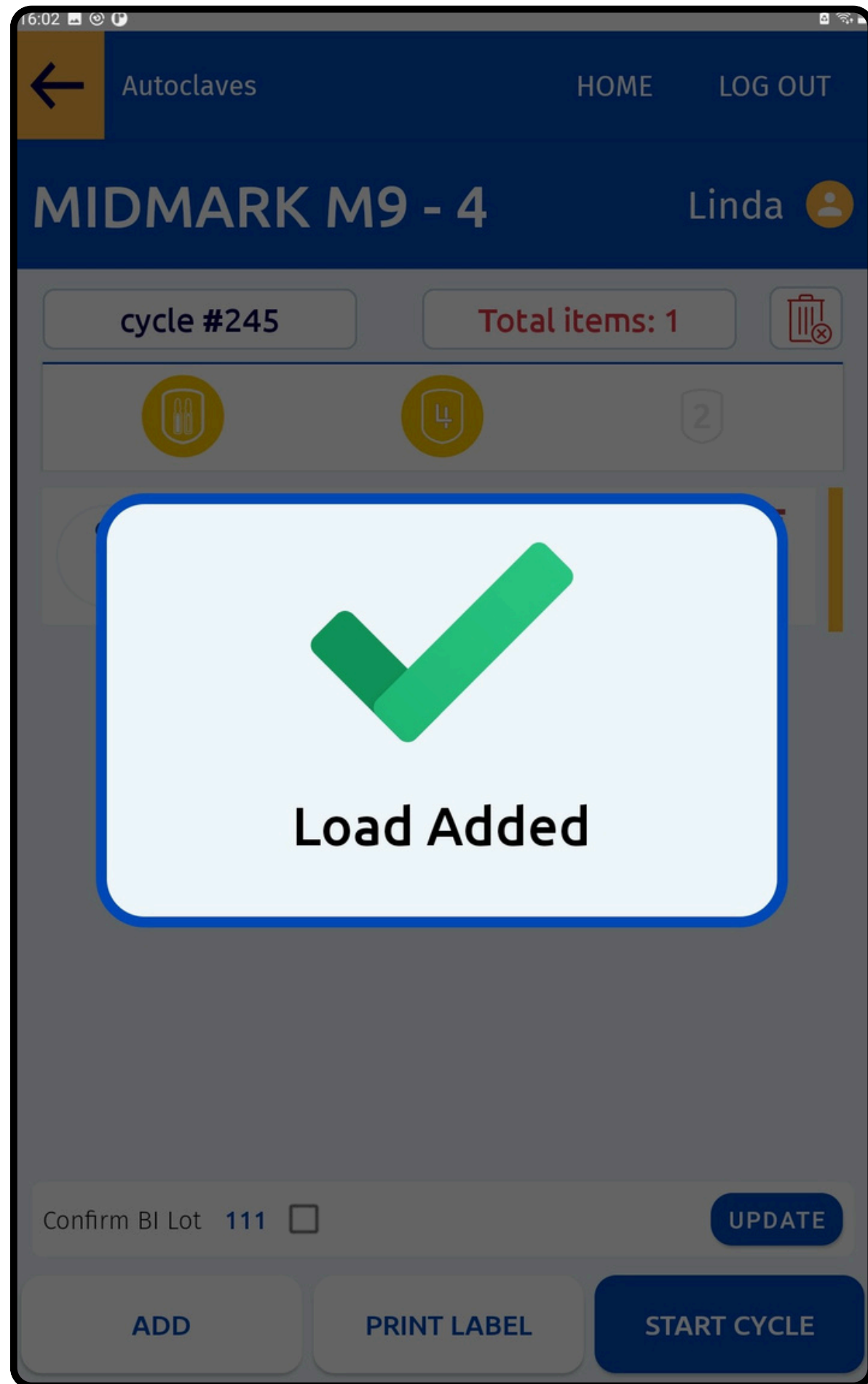



**Successfully Load the  
Autoclave with the  
selected items**

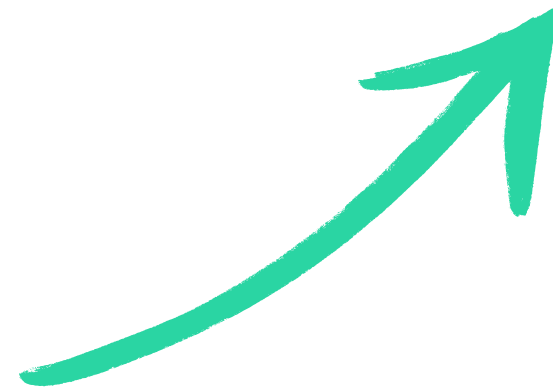




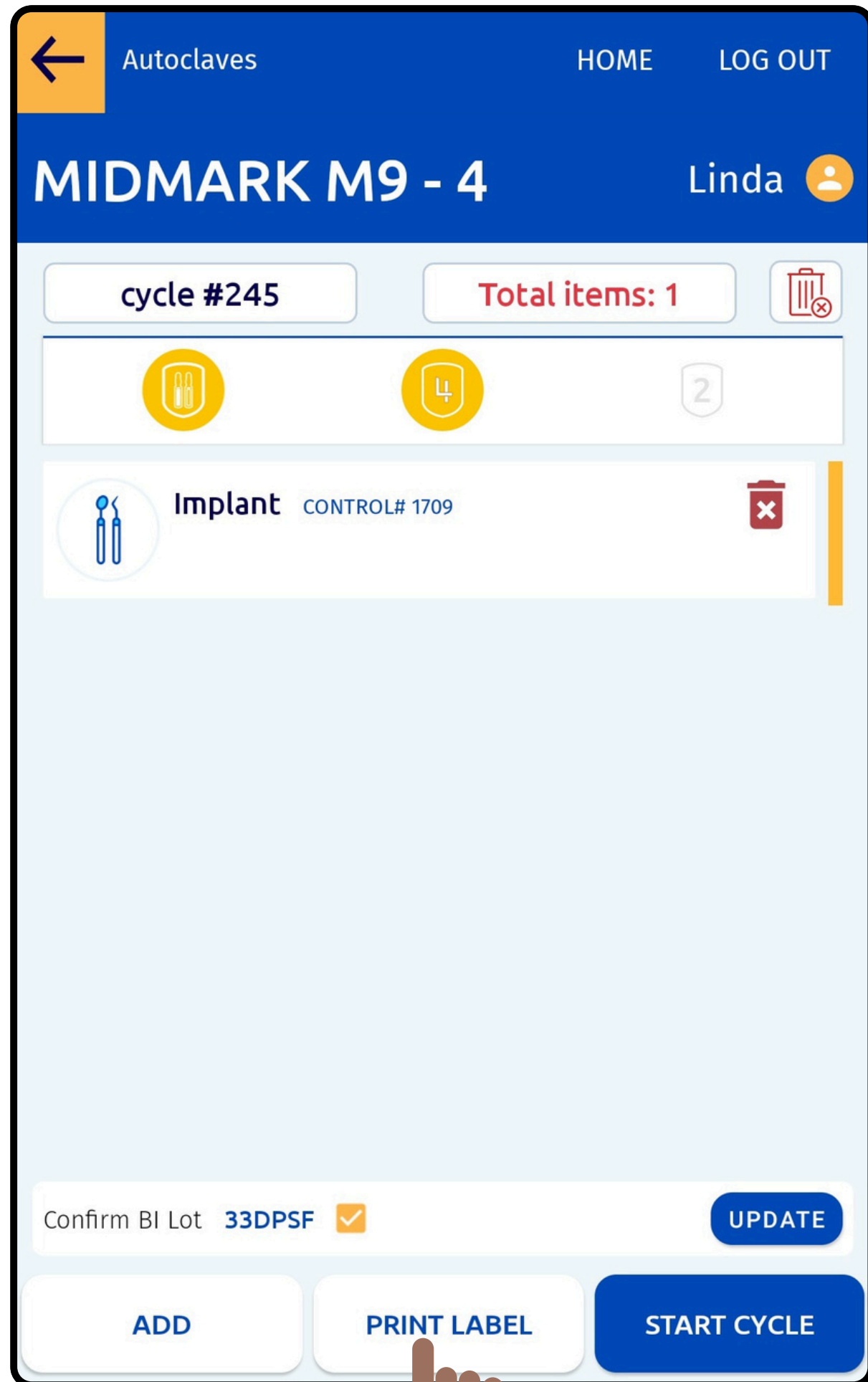


Click on Print Label to  
print label(s)  
for the Items added  
to the Autoclave

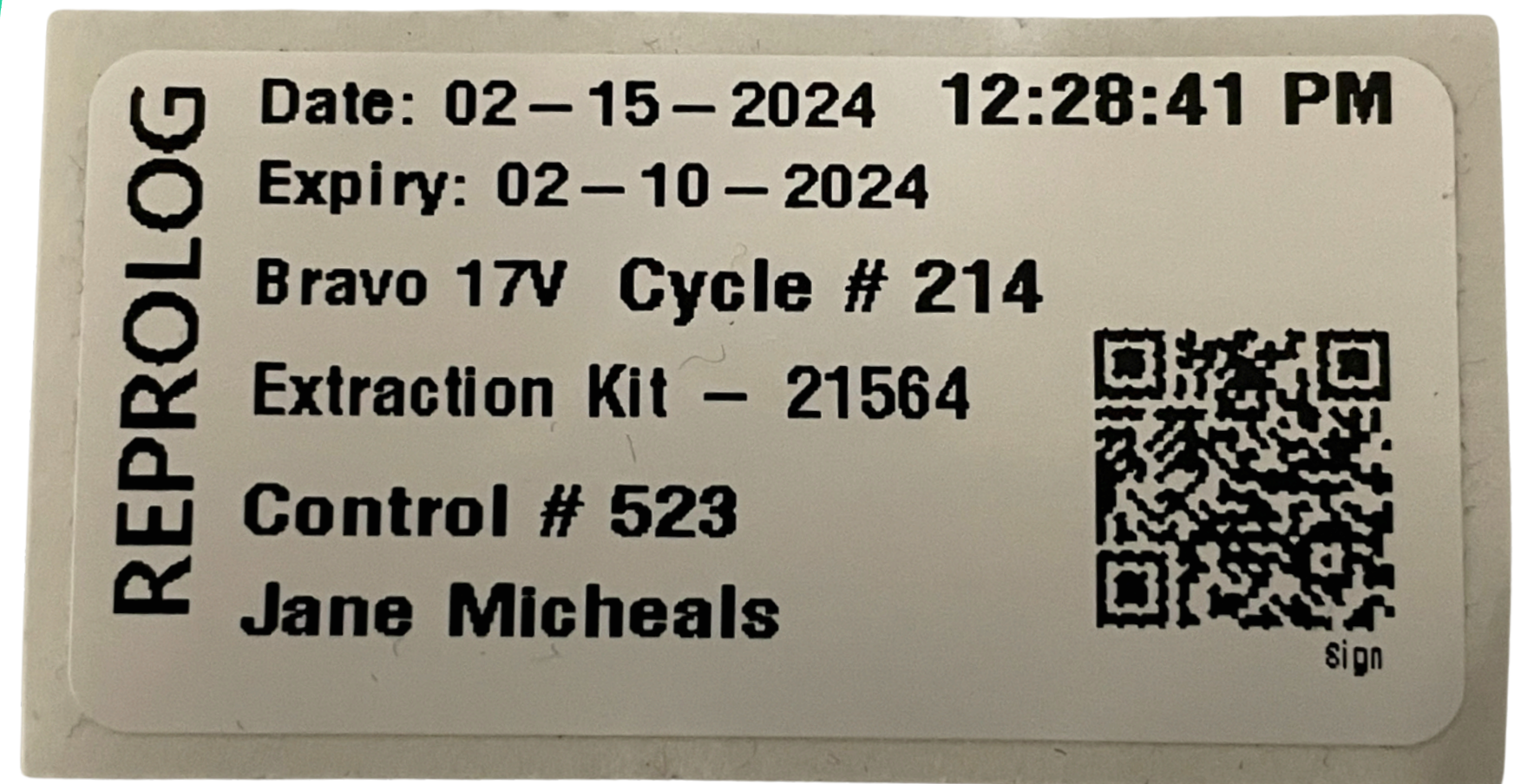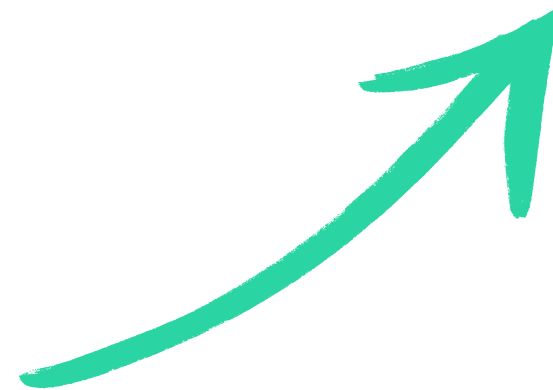




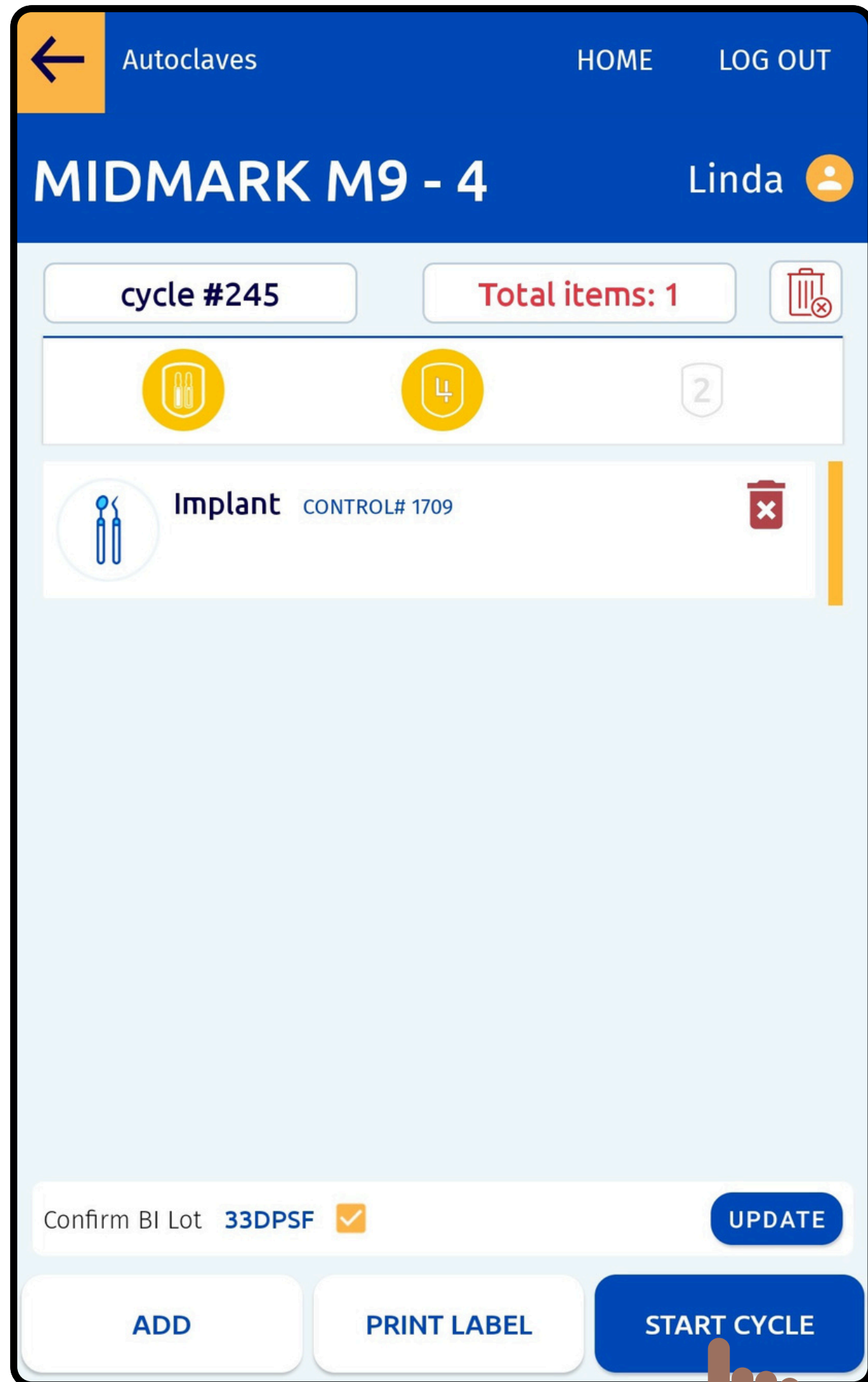


**User can Remove or Add  
Items, Update BI Lot  
Number and Start a Cycle**

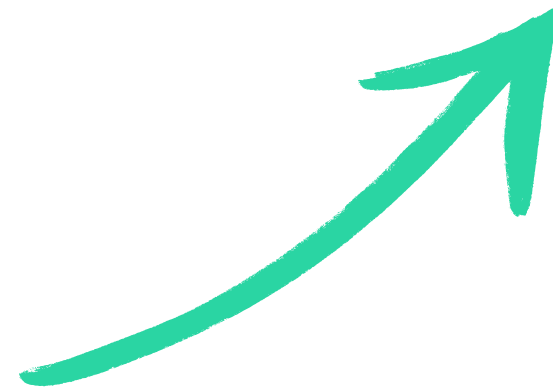



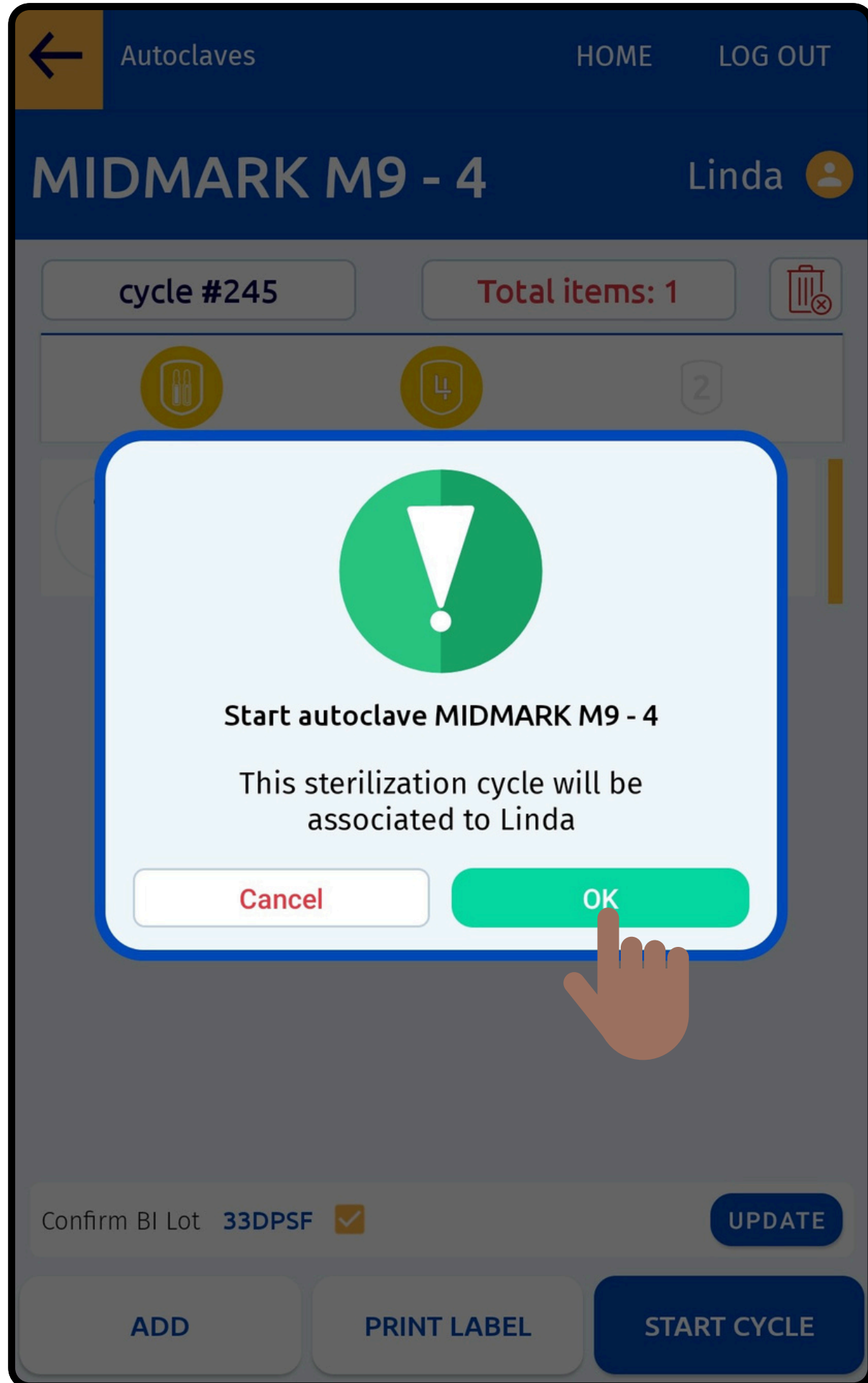


**Confirm the Cycle Details  
for the Load to get started**

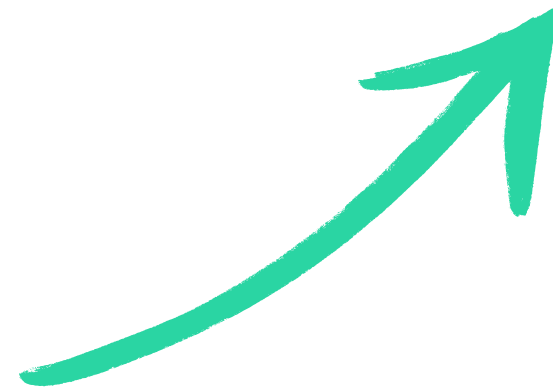








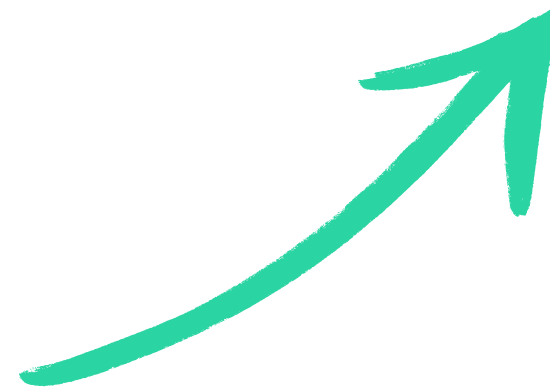
← Main menu HOME LOG OUT

# Sterilize


Linda 

-  **MIDMARK M11 - 1** LOAD TIME : 29:47  
Contains C4 and Total 1 items  
UNWRAPPED
-  **Tuttnauer EZ9Plus - 2** LOAD FINISHED  
Contains C4 and Total 1 items  
WRAPPED INSTRUMENTS, POUCHES
-  **BRAVO 17V - 3** LOAD TIME : 27:32  
Contains total 1 items  
BOWIE DICK
-  **MIDMARK M9 - 4** LOAD TIME : 44:46  
Contains C4, BI and total 1 items  
HANDPIECES
-  **STATIM 5000**  
NO-SERVICE

# View the Autoclave Load Details in Real-Time






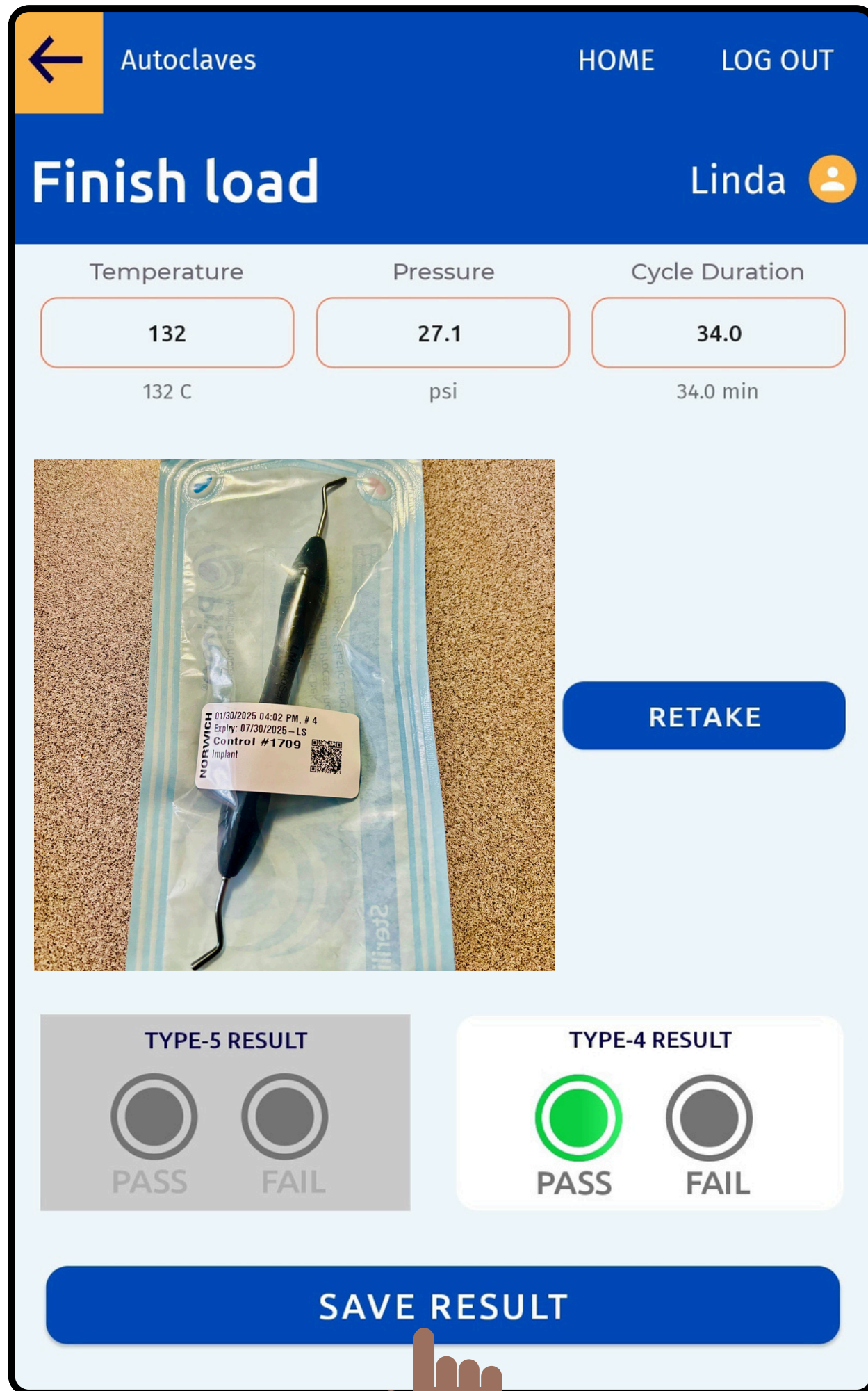

User can Add values for  
Temperature,  
Pressure,  
and Cycle Duration  
and  
Capture the Image



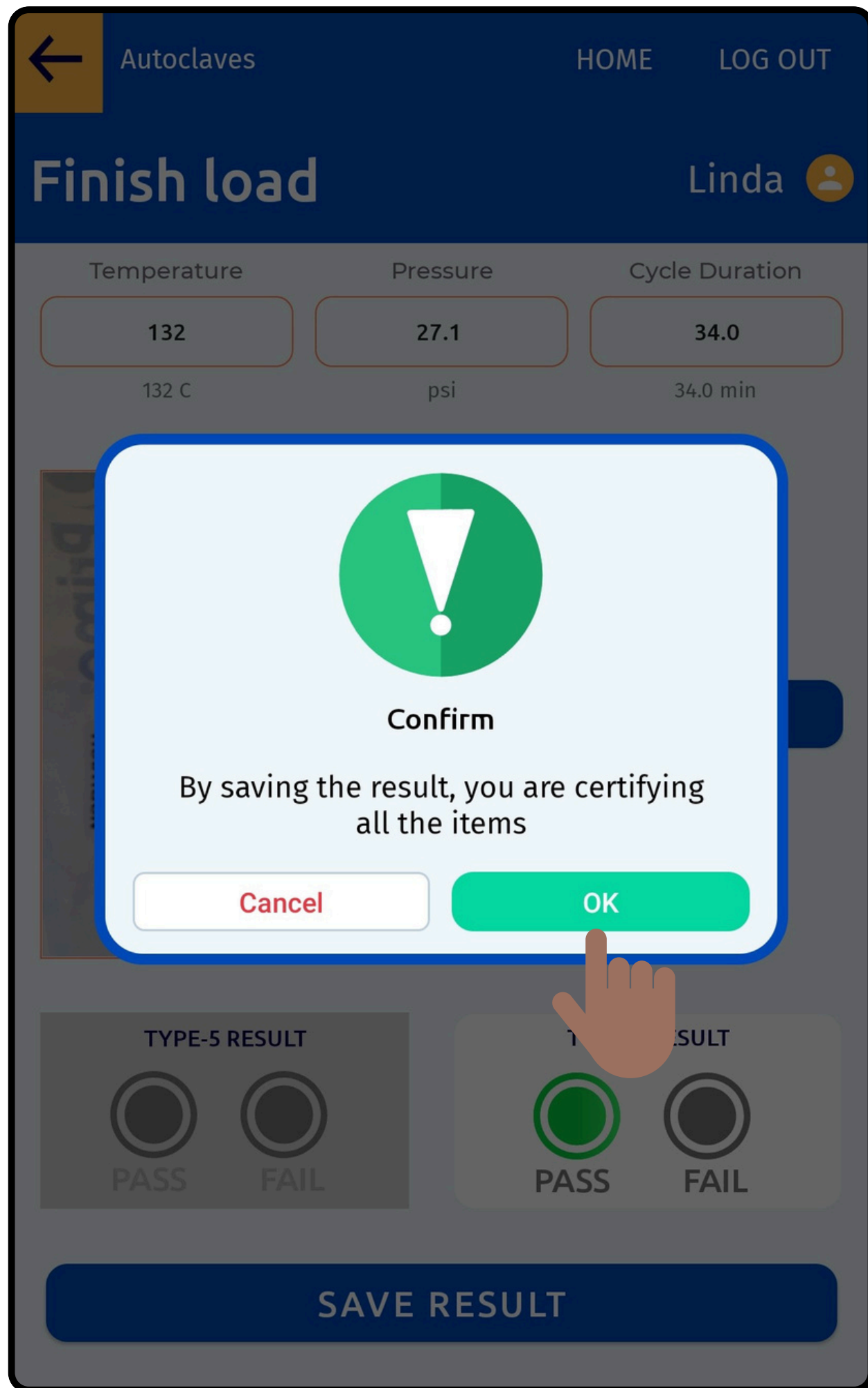

**User can Finish the Load by adding the Result and Certifying the Items**





**Sterilize** 1

Add items to autoclave, start sterilization, certify instruments, record sterilization results.



**Record BI Results** 5

Record the results of Biological Indicator test.



**Use Instruments**

Record items to be used on a patient. Obtain control number here.




**Check Status**

Check the status of a particular item.


GET REPORT

# Select to Record Test Results of Biological Indicator

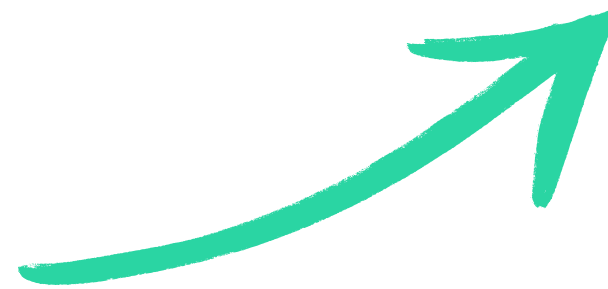














← Main Menu HOME LOG OUT


Select Autoclave Linda 

STORE EXTERNAL SPORE TEST RESULTS

-  **STATIM 5000** Lot 111 C4   
cycle # 228 Unwrapped  
January-20-2025
-  **STATIM 5000** Lot 111 C4   
cycle # 230 Unwrapped  
January-20-2025
-  **Tuttnauer EZ9Plus - 2** Lot 111 C4   
cycle # 221 Wrapped Instruments, Pouches  
January-20-2025
-  **MIDMARK M9 - 4** Lot 111 C4   
cycle # 244 Pouches  
January-20-2025
-  **MIDMARK M11 - 1** Lot 111 C4   
cycle # 200 Packs  
January-20-2025



Click on the Autoclave name to select and Record BI Results

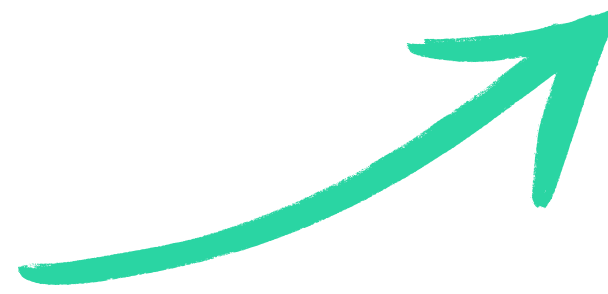




← Main menu HOME LOG OUT

BI Result Linda

AUTOCLAVE CYCLE # BI LOT # DATE  
STATIM 5000 228 111 January-20-2025



RETAKE

CONTROL VIAL

GROWTH NO GROWTH

TEST VIAL


GROWTH NO GROWTH

ADDITIONAL COMMENTS

Add Notes


SAVE BI RESULT

Save the BI Result by capturing an image and recording the growth of Control and Test vials

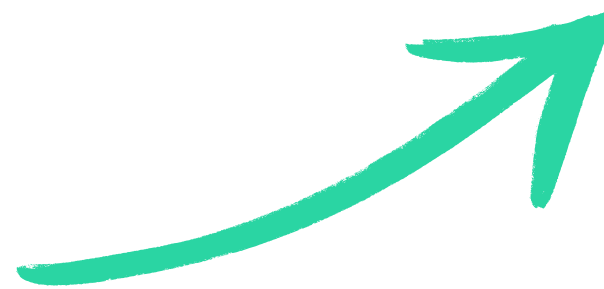



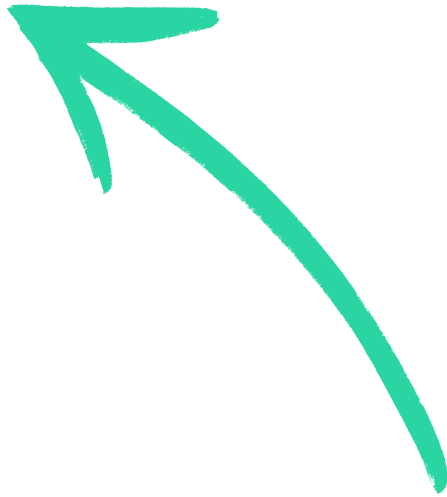

# Click on Use Instruments to Record and Return Items




BELMONT ( AJAX ) LOG OUT

**Main Menu** Linda 



**Sterilize** 2 2


Add items to autoclave, start sterilization, certify instruments, record sterilization results.

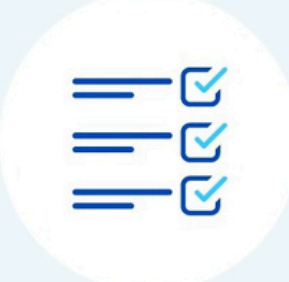

**Record BI Results**


Record the results of Biological Indicator test.


---

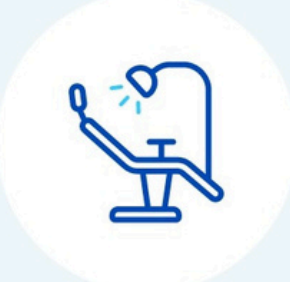

**Use Instruments**

Record items to be used on a patient. Obtain control number here.

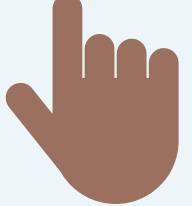





**Check Status**

Check the status of a particular item.


[GET REPORT](#)

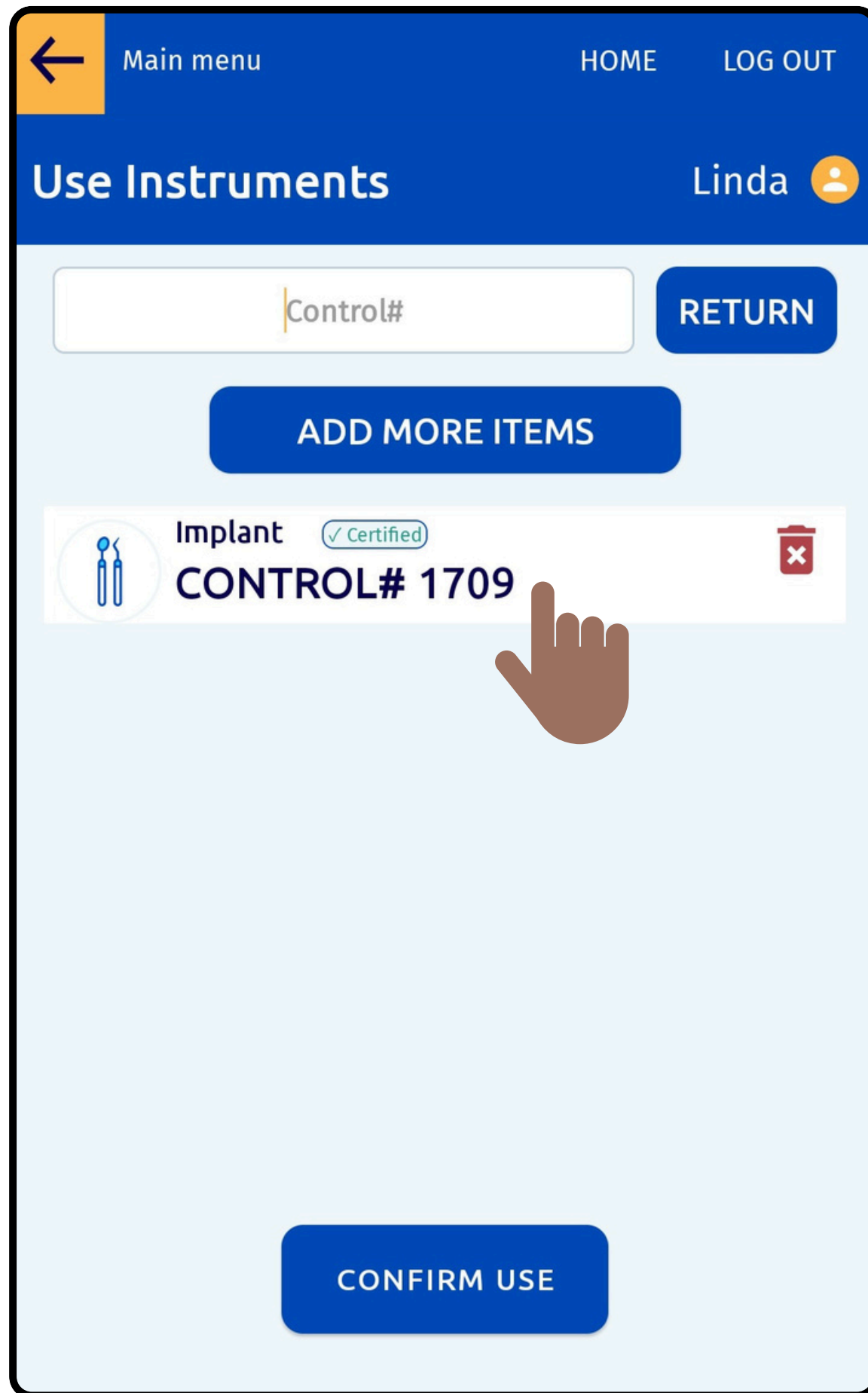


**Enter the Control Number  
to add Items  
for Use or Return**

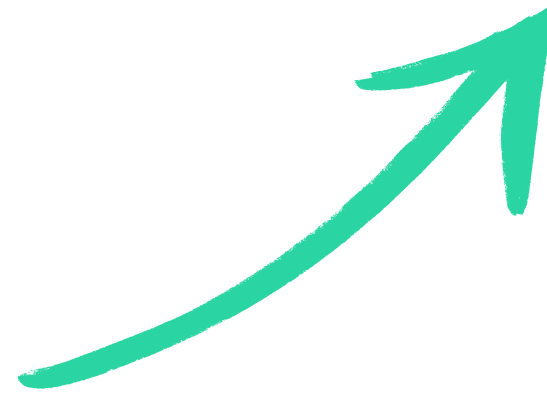



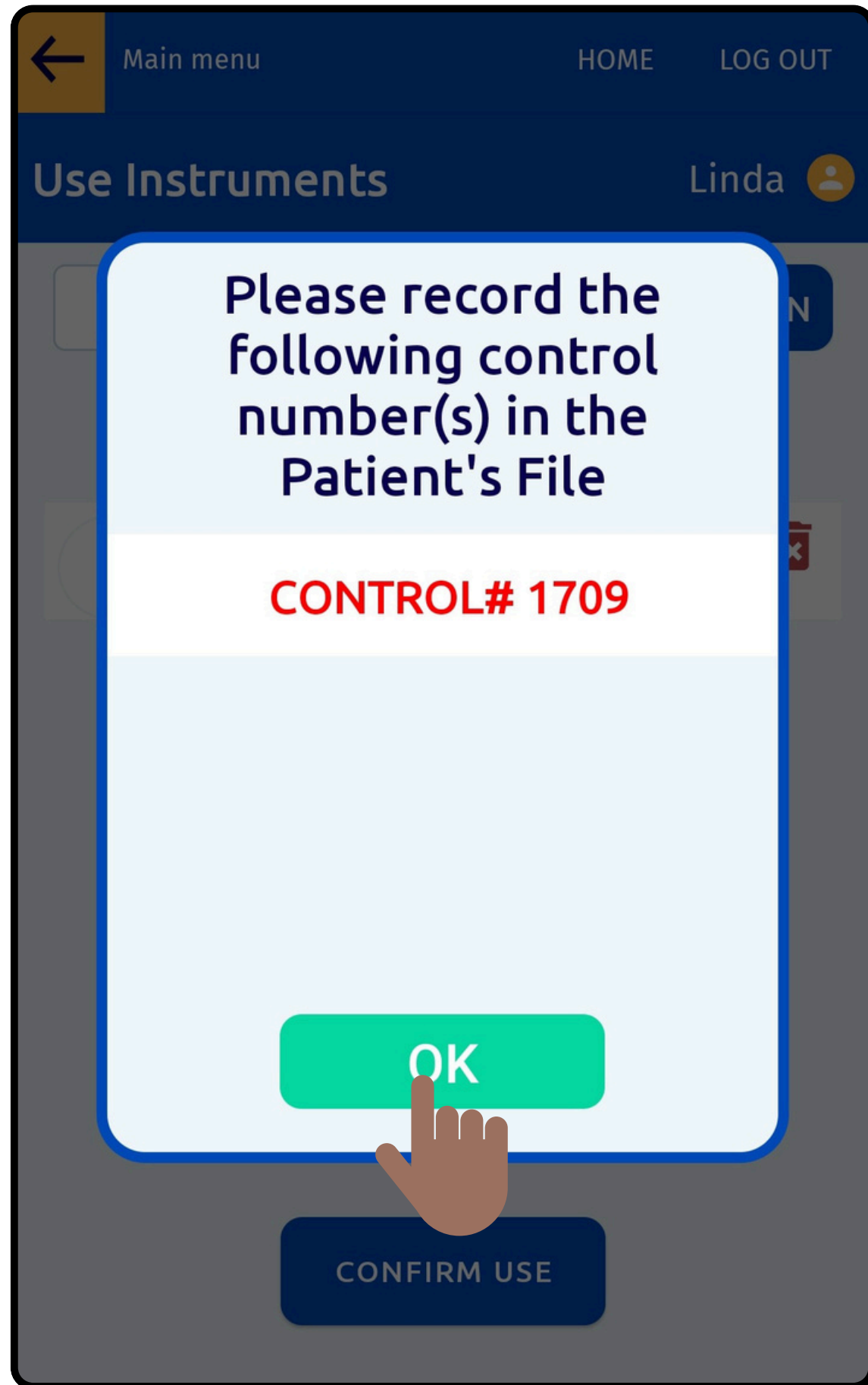

**Click on OK to  
Confirm and Record  
the Control Numbers  
into the Patient File**

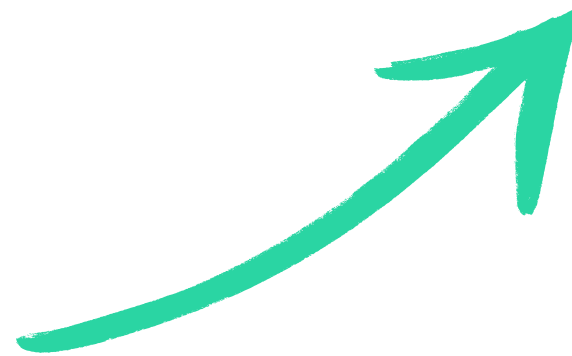



**Sterilize** 2 2

Add items to autoclave, start sterilization, certify instruments, record sterilization results.



**Record BI Results**

Record the results of Biological Indicator test.




**Use Instruments**

Record items to be used on a patient. Obtain control number here.



**Check Status**

Check the status of a particular item.

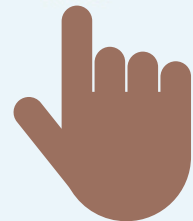


GET REPORT


# Select Check Status to view the status of Items






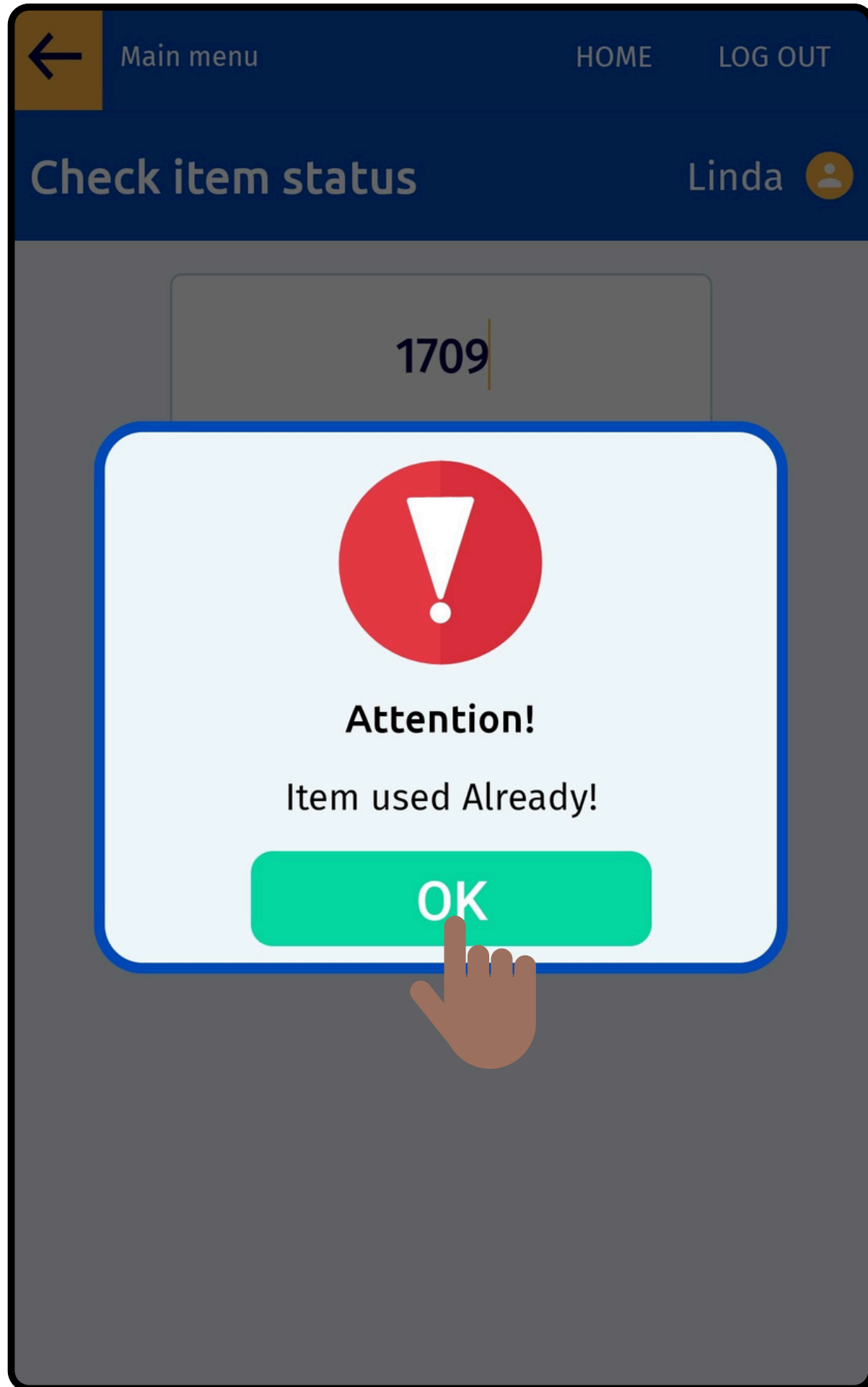

**Enter the Control Number  
to view the Current Status  
of that Item**

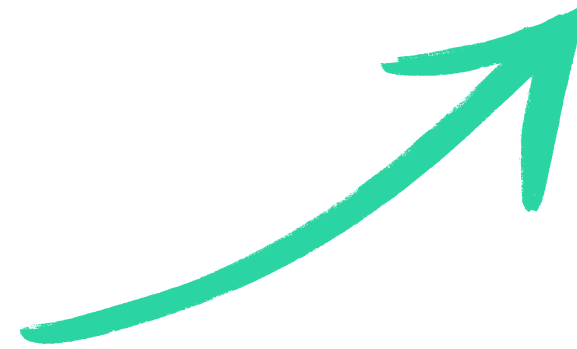

# Select the Get Report button to View and Email the Details of an Item

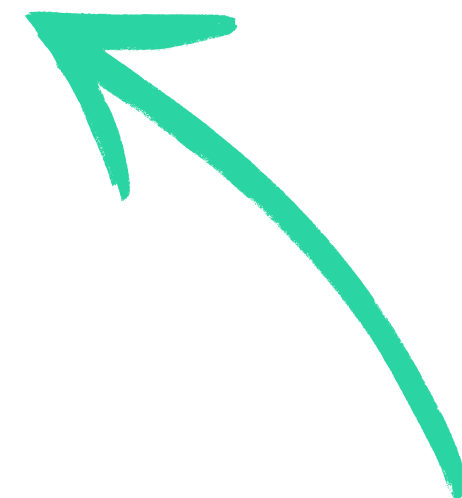

BELMONT ( AJAX ) LOG OUT

**Main Menu** Linda

**Sterilize** 2 2

Add items to autoclave, start sterilization, certify instruments, record sterilization results.

**Record BI Results**

Record the results of Biological Indicator test.

**Use Instruments**

Record items to be used on a patient. Obtain control number here.


**Check Status**

Check the status of a particular item.


**GET REPORT**



# Enter the Control Number and Click Ok

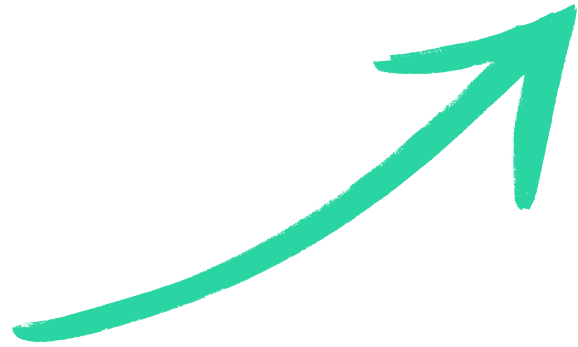

BELMONT ( AJAX ) LOG OUT

Main Menu Linda 

Please enter control

1709

CANCEL OK

Use Instruments  
Record items to be used on a patient. Obtain control number here.

Check Status  
Check the status of a particular item.

GET REPORT

4  
gical

Ad  
steril  
re

The image shows a mobile application interface. At the top, there is a dark blue header with the text 'BELMONT ( AJAX )' on the left and 'LOG OUT' on the right. Below the header, there is a 'Main Menu' section with the name 'Linda' and a user profile icon. The main content area is a light gray background with several icons and text. A modal dialog box is overlaid on the screen, titled 'Please enter control'. It contains a text input field with the number '1709' and two buttons: 'CANCEL' (white with red text) and 'OK' (green with white text). A hand icon is shown clicking the 'OK' button. Below the dialog box, there are two main sections: 'Use Instruments' with a description 'Record items to be used on a patient. Obtain control number here.' and 'Check Status' with a description 'Check the status of a particular item.'. At the bottom, there is a 'GET REPORT' button. On the right side of the dialog box, there is a red square with the number '4' and the word 'gical' below it. On the left side, there is text 'Ad' and 'steril' above 're'.

CONTROL# 1709

Article Name Implant

Loaded By Linda Smith

Sterilized On 30-January-2025

Autoclave Name MIDMARK M9 - 4

Started By Linda Smith


cycle # 245

Used By Linda Smith


Used On 2025-01-30

GO BACK


EMAIL REPORT

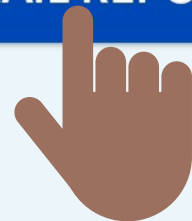


**The user can View the recorded details and Email the report as needed**

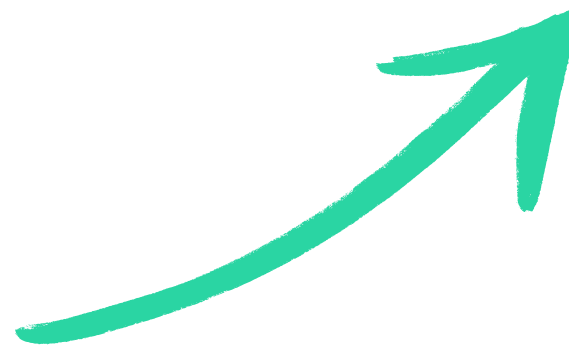



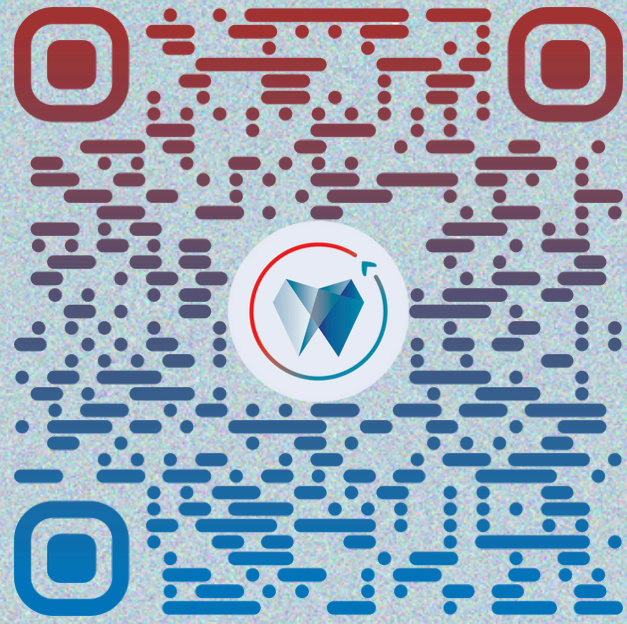

Scan the QR Code  
to book a Demo

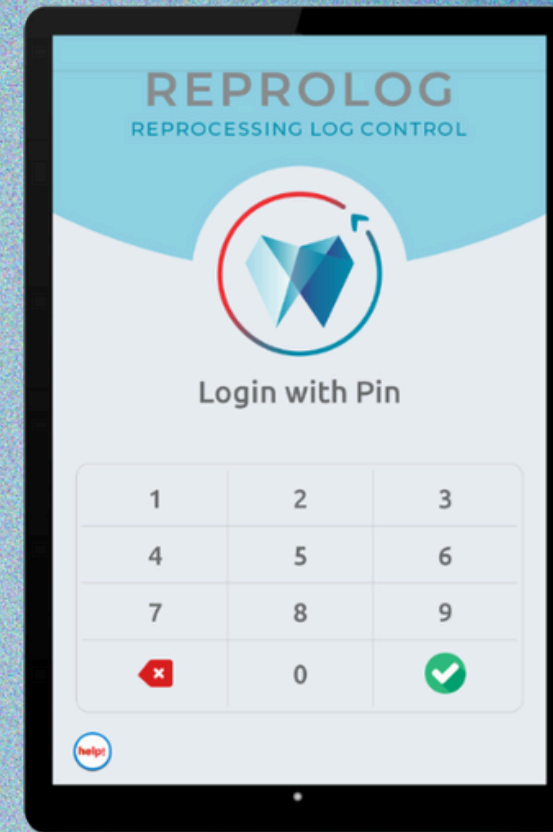

**PHONE**

+1.416-423-2900



**EMAIL**

info@reprolog.net

